

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

SPECIFIC SLING



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL



- CE certified and manufactured in France, the Specific Sling adapts to the morphology of each patient by its multitude of sizes (S, M, L, XL) and is used with all stand-up lifts equipped with a hook system.
- It is used positioned on the lumbar in order to trigger the patient's natural raising scheme (forward traction).
- Its rigid foam padding at the back part level reduces painful pressure and its belt allows transfers in a sitting position

| | | |
|---------|--------|---|
| SWYP-S | JERSEY | ○ |
| SWYP-M | JERSEY | ○ |
| SWYP-L | JERSEY | ○ |
| SWYP-XL | JERSEY | ○ |

Lot

PACKAGING : 1 PIECE

Textile materials in
accordance with the legislation
REACH
on certain stable substances

12 Allée du Piot - ZA Pôle Actif
30660 Gallargues le Montueux
FRANCE


CE
v.04.03.2017

IMPORTANT RECOMMENDATIONS :
In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
These products must not be in direct contact with a wounded skin.

SPECIFIC SLING

SIZE & DIMENSIONS (in cm)

| | Plastron | A | B |
|----|-----------|----|-----|
| S | 70 / 90 | 21 | 78 |
| M | 90 / 115 | 25 | 92 |
| L | 115 / 140 | 29 | 106 |
| XL | 140 / 165 | 33 | 120 |

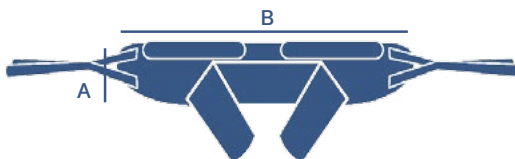

Composition
100 % polyester fabric
40 mm polyester strap - 1450 DaN
100 kg/m³ PE foam
20/25 kg/m³ PE foam
20 mm polyester bias

INSTRUCTIONS FOR USE

Position the bottom of the backrest in the lumbar region in order that the ears of the attachment system are on either side of the person.

Place the fasteners of the backrest under the arms of the person.

Close the plastron.

Hook the two fasteners of the backrest to the stand-up lift hooks.

SETTINGS

The fastening system, with multiple hangers, allows to adjust the position of the person.

CAUTION

Read the instruction manual before using the slings.

The backrest hangers must be hung on the same color.

Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.

Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.

v 04.03/2017



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**