

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

UNIVERSAL ECO SLING (comfort or bath)

COMFORT MODEL

BATH MODEL



Compatible with
removable headrest
(TAMOV).



**THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL**



- CE certified and manufactured in France, the Universal Eco Sling (comfort or bath) is recommended for bed to chair to bed transfers.
- Its U-shape avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.
- The leggings comfort foam reduces the shearing effect and the positioning of hooks at the shoulders level, traversing the entire slings dorsal part, avoids the phenomenon of compression during transfer.
- The bath model is preferred for the bath or shower, because it is made with a high tenacity polyester net in order to evacuate the water.

SU-RPC-TU JERSEY One size fits ○

SU-RP-TU NET One size fits ○

Lot

PACKAGING : 1 PIECE

Textile materials in
accordance with the legislation
REACH
on available substances

12 Allée du Piot - ZA Pôle Actif
30660 Gallargues le Montueux
FRANCE

CE
v.04.03.2017


IMPORTANT RECOMMENDATIONS :
In order to get a maximum efficiency of these sort of product, it is necessary :
- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient
These products must not be in direct contact with a wounded skin.

UNIVERSAL ECO SLING (comfort or bath)

SIZE & DIMENSIONS (in cm)

	A	B	C
One size fits	70	61	92

Composition
100% polyester fabric
40mm polyester strap - 1450 DaN
50kg/m³ PE foam
20mm polyester bias

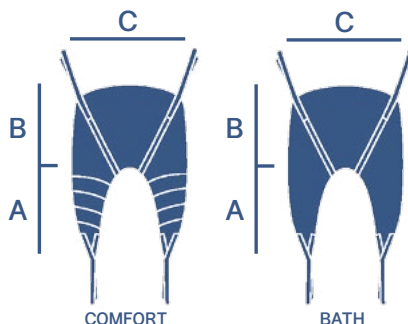

INSTRUCTIONS FOR USE

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lower part of the sling at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Cross these hanging straps. Hang both hanging straps of the back and hanging straps of the thigh supports to the spreader bar.

ADJUSTMENTS :

The different loops on the hanging straps allow various positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 04.03/2017



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**