

*This medical device is intended for patient positioning.*

*This is not a restraint system.*

*Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## Y & PELVIC SLIM BELT



TO PREVENT FORWARD SLIDING  
AND MAINTAIN THE CHEST IN  
UPRIGHT POSITION  
FOR ALL CHAIRS EXCEPT SHELLS



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL



### HOME CARE RANGE

|          |                  |         |                       |
|----------|------------------|---------|-----------------------|
| CSYP-80  | WAIST 60/80 cm   | Size S  | <input type="radio"/> |
| CSYP-95  | WAIST 75/95 cm   | Size M  | <input type="radio"/> |
| CSYP-110 | WAIST 90/110 cm  | Size L  | <input type="radio"/> |
| CSYP-130 | WAIST 110/130 cm | Size XL | <input type="radio"/> |



### HOSPITAL RANGE

|             |                  |         |                       |
|-------------|------------------|---------|-----------------------|
| CSYP-HP-80  | WAIST 60/80 CM   | Size S  | <input type="radio"/> |
| CSYP-HP-95  | WAIST 75/95 CM   | Size M  | <input type="radio"/> |
| CSYP-HP-110 | WAIST 90/110 CM  | Size L  | <input type="radio"/> |
| CSYP-HP-130 | WAIST 110/130 CM | Size XL | <input type="radio"/> |

Lot

PACKAGING : 1 PIECE

Textile materials in  
accordance with the legislation  
**REACH**  
on stable substances

 12 Allée du Piot - ZA Pôle Actif  
30660 Gallargues le Montueux  
FRANCE

**CE**  
v.04.03.2017

**IMPORTANT RECOMMENDATIONS :**

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

## Y & PELVIC SLIM BELT

### Compositions :

#### HOME CARE RANGE

Combination of polyurethan foam and jersey/  
cotton polyester  
Jersey 100% polyester fabric  
100% cotton bias  
100% polyester strap  
Nylon buckle  
Thermo compressed foam

#### HOSPITAL RANGE

Combination of polyurethan foam and on  
both faces  
Jersey/PU coating  
100% polyester bias  
100% polyester strap  
Nylon buckle  
Thermo compressed foam

### INSTRUCTIONS FOR USE

- Position the pelvic part on the seat of the chair and let the buckle go out towards the back of the chair.
- Let the patient sit on the chair.
- Position the belt around the patient's waist.
- Close the belt using the velcro strap.
- Place the chest belt being careful that the shoulders pads are positioned on the patient's shoulders.
- Pass the strap of the back part of the belt behind the backrest of the chair.
- Before closing it and without adjusting it, check that the back strap has a female buckle looking downwards and 2 female buckles looking upwards.
- Close the 4 rear buckles, adjust

v 04.03/2017



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.**