

*This medical device is intended for patient positioning.*

*This is not a restraint system.*

*Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## FOOT SUPPORT



**HOME CARE RANGE**



**SUPPORTS THE FOOT  
ON THE WHEELCHAIR FOOTREST.**

LMDP

BY PAIR



**THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL**



**HOSPITAL RANGE**



**SUPPORTS THE FOOT  
ON THE WHEELCHAIR FOOTREST.**

LMDP-HP

BY PAIR



### Description

The Foot Support upholds the foot on the wheelchair footrest due to its velcro strap fixation under and on the side of the footrest.

Lot

PACKAGING : 1 PIECE

Textile materials in  
accordance with the legislation  
**REACH**  
on undesirable substances

12 Allée du Piot - ZA Pôle Actif  
30660 Gallargues le Montueux  
FRANCE

**CE**  
v.04.03.2017

**IMPORTANT RECOMMENDATIONS :**

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

## FOOT SUPPORT

### Compositions :

#### HOME CARE RANGE

Combination of polyurethane foam  
and jersey cotton/Polyester fabric

100% cotton bias

100% polyester bias

100% polyester velcro strap

100kg/m3 PU foam

#### HOSPITAL RANGE

Combination of polyurethane foam  
and on both faces jersey/PU coating

100% cotton bias

100% polyester bias

100% polyester velcro strap

100kg/m3 PU foam

### Instructions for use

TO BE USED ONLY FOR THE INDICATED USE:

#### UPHOLDS THE FOOT ON THE WHEELCHAIR FOOTREST

Fix each foot support on the wheelchair footrest with the velcro strap

Place each patient's foot into a foot support

Close the foot support on the patient's foot by using the velcro strap



v 04.03/2017



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.