

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

BARIATRIC SLING



THIS MEDICAL
DEVICE MUST BE
IMPLEMENTED BY
A TRAINED HEALTH
PROFESSIONAL



- CE certified and manufactured in France, the Bariatric Sling is specifically recommended for bariatric patients transfers.
- The leggings in 3D fabric, the back part in rigid foam and the integrated headrest allow to protect the spine during transfers.
- Its U-shape avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.

SG-GLT-2XL	JERSEY	2XL ○
SG-GLT-3XL	JERSEY	3XL ○
SG-GLT-4XL	JERSEY	4XL ○

Lot

PACKAGING : 1 PIECE

Textile materials in
accordance with the legislation
REACH
on undesirable substances

12 Allée du Piot - ZA Pôle Actif
30660 Gallargues le Montueux
FRANCE

CE
v.04.03.2017

IMPORTANT RECOMMENDATIONS :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

BARIATRIC SLING

SIZE & DIMENSIONS (in cm)

	A	B	C
2XL	100	92	112
3XL	123	96	130
4XL	146	101	148

Composition

100% Polyester Fabric

Three-dimensional polyester fabric


Sling 100% Polyester 40 mm - 1450 DaN

50Kg/M3 PE foam

100Kg/M3 PE foam

40 mm polyester Bias

20 mm polyester Bias

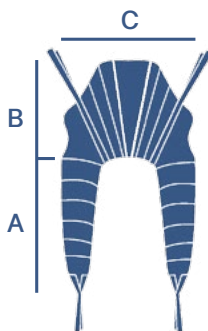

INSTRUCTIONS FOR USE

Do not use this sling if the size is not suitable for the patient.

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreadbar :

- 2 on both sides of the back
- 1 at each end of the thigh support

Position the bottom of the back rest at the level of the coccyx, so that the suspension of the attachment system are on either side of the person. By slightly raising the right leg of the person, pass the right thigh support under the right thigh. Do the same for the left leg. Both fastening devices for thigh supports are located between the legs of the person. Pass both systems through one another. Hook the two fasteners of the back rest and the two thigh fasteners to the spreader bar of the lift.

ADJUSTMENT

The different loops on the hanging straps allow various positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 04.03/2017



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.