

*Prior to each use, it is important to check the status of the device:  
seams, state of the straps and of the fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## UNIVERSAL ECO TOILET SLING



Compatible with  
boned headrest  
(TABAL).



**THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL**



- CE certified and manufactured in France, the Universal Eco Toilet Sling is recommended for toilet and daily change transfers.
- Its extended U-shape completely frees the patient's buttocks thus facilitating the positioning to the toilet, and improving the installation and removal of the sling.
- This sling is not recommended for patients with very low tone.

**STO-RP-TU JERSEY**      One size fits    ○

Lot

PACKAGING : 1 PIECE

Textile materials in  
accordance with the legislation  
**REACH**  
on undesirable substances

12 Allée du Piot - ZA Pôle Actif  
30660 Gallargues le Montueux  
FRANCE

**CE**  
v.04.03.2017

#### IMPORTANT RECOMMENDATIONS :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

## UNIVERSAL ECO TOILET SLING

### SIZE & DIMENSIONS (in cm)


	A	B	C
One size fits	75	25	108

#### Composition

Jersey 100 % polyester

Sangle polyester 40 mm - 1450 DaN

Mousse PE 50 kg/m3

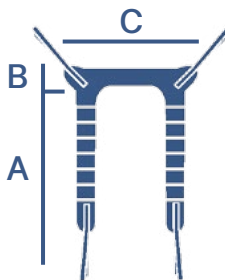

### CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 04.03/2017



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.