

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

STANDARD SLING



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL



- CE certified and manufactured in France, the Standard Sling adapts to the morphology of each patient by its multitude of sizes (S, M, L, XL) and is used with all stand-up lifts equipped with a hook system.
- It is placed under the armpits based on the device used, due to its full padding under the arms which distributes traction on the patient and avoids painful pressure.

SAPVR-S	JERSEY	○
SAPVR-M	JERSEY	○
SAPVR-L	JERSEY	○
SAPVR-XL	JERSEY	○

Lot

PACKAGING : 1 PIECE

Textile materials in
accordance with the legislation
REACH
on stable substances

12 Allée du Piot - ZA Pôle Actif
30660 Gallargues le Montueux
FRANCE

CE
v.04.03.2017

IMPORTANT RECOMMENDATIONS :

In order to get a maximum efficiency of these sort of product, it is necessary :


- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

STANDARD SLING

SIZE & DIMENSIONS (in cm)

	Plastron	A	B
S	70 / 90	21	78
M	90 / 115	25	92
L	115 / 140	29	106
XL	140 / 165	33	120

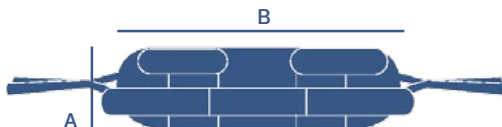

Composition

- 100 % polyester fabric
- 40 mm polyester strap - 1450 DaN
- 50 kg/m³ PE foam
- 20/25 kg/m³ PE foam
- 20 mm polyester bias

INSTRUCTIONS FOR USE

Position the bottom of the backrest around the shoulder blades in order that the ears of the attachment system are on either side of the person.

Place the fasteners of the backrest under the arms of the person, so that the beads are under the armpits.

Close the plastron.

Hook the two fasteners of the backrest to the stand-up lift hooks.

SETTINGS

The fastening system, with multiple hangers, allows to adjust the position of the person.

CAUTION

Read the instruction manual before using the slings.

The backrest hangers must be hung on the same color.

Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.

Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.

v 04.03/2017



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**