

PATIENT LIFTS SLINGS RANGE USER GUIDE

Straps Positioning



Low straps crossed between the legs



Low straps crossed under the legs



Low straps uncrossed between the legs

Size Chart

Weight / Sizes	1m50	1m55	1m60	1m65	1m70	1m75	1m80	1m85	1m90 + 1m90
40 kg	S	S	S	S	S	S	M	M	M
45 kg	S	S	S	S	S	S	M	M	M
50 kg	S	S	S	S	S	S	M	M	M
55 kg	S	S	S	S	S	S	M	M	M
60 kg	M	M	M	S	S	S	M	M	M
65 kg	M	M	M	S	S	S	M	M	M
70 kg	M	M	M	M	M	M	M	M	M
75 kg	L	L	M	M	M	M	M	M	M
80 kg	L	L	L	L	M	M	M	M	M
85 kg	L	L	L	L	M	M	M	M	M
90 kg	L	L	L	L	L	L	M	M	M
95 kg	XL	XL	XL	XL	L	L	M	M	M
100 kg	XL	XL	XL	XL	L	L	L	L	L
110 kg	XL	XL	XL	XL	L	L	L	L	L
120 kg	XL	XL	XL	XL	XL	XL	L	L	L
130 kg	XL	XL	XL	XL	XL	XL	XL	XL	XL
160 kg	2 XL	2 XL	2 XL	2 XL	2 XL	2 XL	XL	XL	XL
190 kg	3 XL	3 XL	3 XL	3 XL	2 XL	2 XL	XL	XL	XL
220 kg	3 XL	3 XL	3 XL	3 XL	3 XL	2 XL	2 XL	XL	XL
250 kg	4 XL	4 XL	3 XL	3 XL	3 XL	3 XL	3 XL	2 XL	2 XL
280 kg	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	3 XL	3 XL	3 XL
320 kg	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL

Sizing is an important element when deciding on a sling.

In order to choose the right sling, the caregiver shall consider the 3 following factors with the patient:

- height
- waist size
- thighs size

Choosing the appropriately sized sling will improve the patient's comfort and safety during transfer.

For this reason, a size colour-code of the edges has been put in place in order to quickly identify the size of each sling:

- to the size S corresponds the Yellow colour
- to the size M corresponds the Red colour
- to the size L corresponds the Green colour
- to the size XL corresponds the Blue colour
- to the size 2 XL, 3XL, 4XL corresponds Orange colour
- to the size TU (one size fits) corresponds the Black colour

Headrests



TAMOV



TABAL



TAMAX

HEADREST

The addition of a headrest helps to take in charge the patient's head and protects the cervical.

3 models are available :

- TAMOV _ removable headrest
- TABAL _ boned headrest
- TAMAX _ removable headrest for XXL slings