



*This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.*

## SIMPLE BIRDY BELT

FOR ALL ARMCHAIRS - SPECIAL SHELL



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL




TO USE ONLY FOR THE INDICATED USE:

**PREVENT SLIDING  
FROM THE BASIN TO THE  
FRONT**

Maximum useful scope: + 220 cm  
Buckle: 50 mm

It can be set up with the patient seated and avoids any pelvic constriction.

- Available in 4 sizes S, M, L, XL
- Waist (in cm)

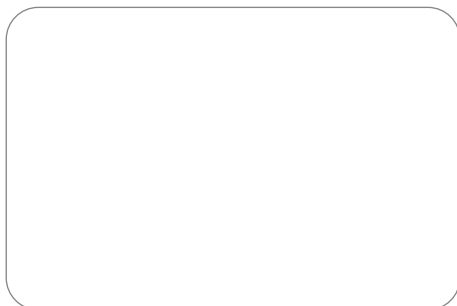

CBR2HP-PF

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




# SIMPLE BIRDY BELT

HOSPITAL RANGE  
Coated polyester jersey  
doubled polyurethane  
polyurethane foam  
Bias 100% polyester  
100% polyester straps  
Nylon buckles

## INSTRUCTIONS FOR USE

to use only for the indicated use:  
**PREVENT BASIN SLIDING FORWARD**

1. Slide the Birdy belt into the back of the patient at the lumbar level and adjust passersby on each hip.
2. Pass the end of the belt under the leg slightly raised.
3. Replicate the operation on the second thigh.
4. Pass each loop through the loop related.
5. Close the buckle on the back of the chair.

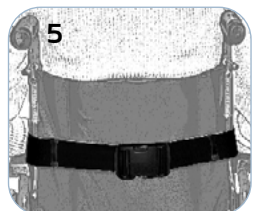

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.