



*This medical device is intended for patient positioning.
This is not a restraint system.*

Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.

LARGE ABDOMINAL BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL




**TO COMPENSATE
SLACKENING
OF ABDOMINAL MUSCLES
TO POSITION IN THE BED**

Maximum useful area: + 230 cm
Height of the belt: + 18 cm
Buckle: 50 mm

Its large contact surface prevents any abdominal constriction.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm
- Waist (in cm)

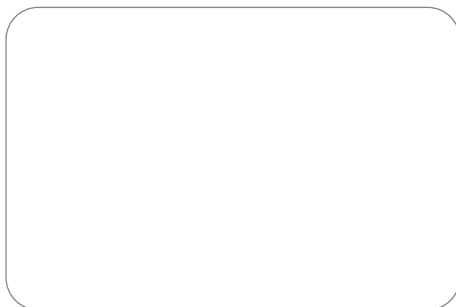

CCA2HP-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



LARGE ABDOMINAL BELT

HOSPITAL RANGE
Combination of polyurethan
foam and on both faces
Jersey/PU coating
100% polyester bias
100% polyester strap
Nylon buckle

INSTRUCTIONS FOR USE TO COMPENSATE SLACKENING OF ABDOMINAL MUSCLES TO POSITION IN THE BED

- Place the belt around the patient's waist, smooth face outwards.
- Put the pointed end through the slit.
- Pass the strap around the backrest of the chair or around the mattress.
- Adjust the length of the straps through the Nylon buckles.
- Used in bed, the degree of restraint varies depending on the setting :
 - Slit on the stomach : kept on the back without any possibility for the patient to reach the buckles.
 - Slit in the back : possibility to move laterally.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.