



*This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.*

## SIMPLE SLIM BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL




TO BE USED ONLY FOR THE  
INDICATED USE :  
**TO PREVENT FROM  
FALLING**

Maximal strap length : + 230 cm  
Belt height : + 11 cm  
Buckle : 50 mm

This simple and robust system is common to all SLIM belts.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm
- Waist (in cm)

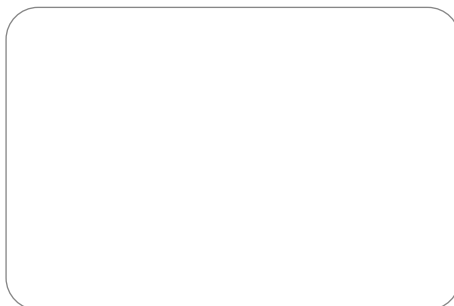

CCS2HP-PF

### Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



## SIMPLE SLIM BELT

HOSPITAL RANGE  
Coated polyester jersey  
doubled polyurethane  
polyurethane foam  
Bias 100% polyester  
100% polyester straps  
Nylon buckles

### INSTRUCTIONS FOR USE

**USE ONLY FOR THE PURPOSE INDICATED: PREVENTING FALLS**

- Position the belt at the waist of the person.
- Close the belt using the Velcro closure.
- Pass the straps behind the back of the chair.
- Close the nylon buckle and adjust the length of the straps, making sure not to compress the patient's abdomen.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.