



This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

BUST SUPPORT BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL




**TO MAINTAIN THE CHEST
IN UPRIGHT POSITION**

Maximum useful area: + 190 cm
Buckle: 40 mm

It provides reinforced comfort of the bust.

- Declined in 3 sizes S, M, L corresponding to the sizes of clothing.
- Maximum useful area: 190 cm

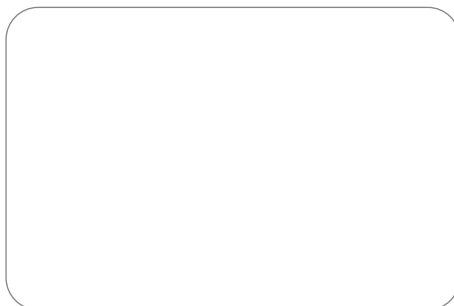

CMB2HP-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



BUST SUPPORT BELT

HOSPITAL RANGE
JERSEY POLYESTER ENDUIT DE
POLYURETHANE DOUBLE DE MOUSSE
POLYURETHANE
BIAIS 100% POLYESTER
SANGLES 100% POLYESTER
BOUCLES NYLON

INSTRUCTIONS FOR USE

- Put on the breast support through the head.
- Pass the belt strap through the loop which is situated at the end of the support braces and grasp the backrest of the chair.
- Adjust length of straps depending on the needs.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.