



Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

PELVIC SUPPORT BELT

FOR ALL CHAIRS

Complies with the ANSM's recommendation concerning awareness of the risk of suffocation when securing a patient using a bed rest device.




THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL



USE :

**PREVENT PATIENTS FROM
FALLING, TO COMPENSATE
SLACKENING OF ABDOM-
INAL MUSCLES AND TO
POSITION IN BED.**

The Bed Support Belt is designed to prevent falls in bed.
It gives the patient a secure positioning.
Compatible with the NAUSI-SURE system (magnetic closure).

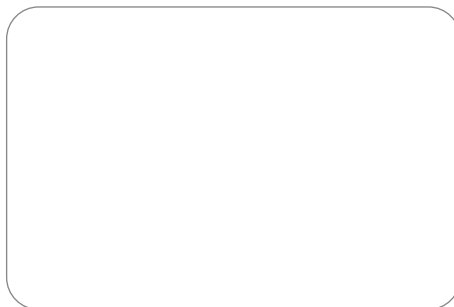

CMLP2HP-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
 - to get the best possible adjustment to the patient
- These products must not be in direct contact with a wounded skin.



PELVIC SUPPORT BELT

Complies with the ANSM's recommendation concerning awareness of the risk of suffocation when securing a patient using a bed rest device.

HOSPITAL RANGE
Combination of polyurethan foam
and jersey/cotton polyester fabric
100 % polyester jersey
100 % polyester bias
100 % polyester strap
Nylon buckle

INSTRUCTIONS FOR USE to use only for the indicated use:

PREVENTING FALLS, SUPPORTING ABDOMINAL RELEASE AND POSITION IN THE BED

- Put the waistband around the waist of the person, the slit on the top (photo 1).
- Pass the strap of the narrow end into the slot of the belt.



Photo 1



Photo 2



Photo 3

- Pass the two straps of the belt part around the fixed amounts of the bed base (use bed-photo 2) or behind the back of the chair.
- Connect the 2 straps of the belt by passing them on the belly of the patient with the click-clack loop and adjust the tension (use bed). This strap can be secured with the NAUSI-SURE device (photo 3).

- Pass the first leg under the thigh on the same side and pass the strap in the loop on the same side.

- Replicate the operation for the other leg.

- Pass each leg strap around the fixed post of the bed base and connect them with the click-clack loop by passing them over the patient (use bed-photo 4-5) or behind the back (use chair). Adjust the voltage.



Photo 4



Photo 5

BED USE IMPORTANT:

- MAKE SURE THAT THE POSITIONING OF THE DEVICE DOES NOT INTERFERE WITH THE USE OF BUSTING (RISK OF COMPRESSION).
- WELL POSITION THE DEVICE AT THE FIXED PART OF THE ARTICULATED BED BASE.
- CLOSE THE BARRIERS AND PUT THE BED IN THE LOWER POSITION.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL OR THE USER MUST BE EDUCATED TO SET IT UP.