



This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

PELVIC SLIM BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL




**PREVENT SLIDING
FROM THE BASIN TO THE
FRONT**

Maximum useful area: + 230 cm
Belt height and width
pelvic part: + 11 cm
Buckle: 50 mm

It has been designed to achieve increased pelvic comfort in the chair.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm (the backrest and seat of the chair must have a separation space).
- Waist (in cm)

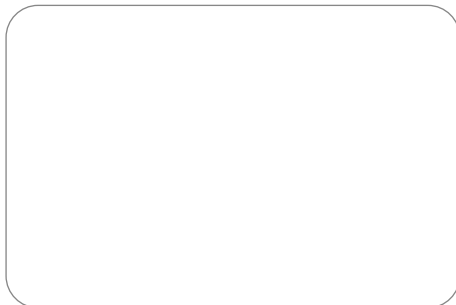

CSP2HP-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



PELVIC SLIM BELT

HOSPITAL RANGE

Polyurethane coated polyester jersey
lined with polyurethane foam
Bias 100% polyester
100% polyester straps
Nylon buckles

INSTRUCTIONS FOR USE

**to use only for the indicated use:
PREVENT BASIN SLIDING FORWARD**

- Position the pelvic girdle on the chair seat, the strap underneath.
- Pass the loop of the pelvic part between the seat and the back of the chair.
- Sit the patient on the chair and raise between the legs the waistband.
- Adjust the belt around the patient's abdomen.
- Gird the patient and the back of the chair with the lap strap.
- Close the loop and adjust the length, being careful not to compress the abdomen.
- Close the pelvic loop and adjust its length (see front).

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.