



*This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.*

## Y SLIM BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL




**TO MAINTAIN THE CHEST  
IN UPRIGHT POSITION**



Maximum useful area: + 230 cm  
Height of the belt: + 11 cm  
Width strap: + 48 mm  
Buckle: 50 mm

The shoulder heights are separately adjustable to help position and compensate for a wrong position.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm
- Waist (in cm)

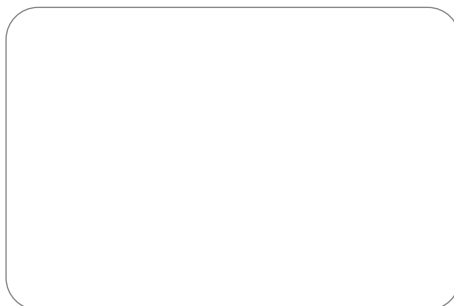

CSY2HP-PF

v 05.12/2020

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient

- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



# Y SLIM BELT

## HOSPITAL RANGE

Combination of polyurethan foam and on both faces jersey/PU coating.

100% polyester bias

100% polyester strap

nylon buckle

thermo compressed foam

## INSTRUCTIONS FOR USE

### TO MAINTAIN THE CHEST IN UPRIGHT POSITION

- Place the shoulder belt being careful that the shoulders pads are positioned on the patient's shoulders, buckles behind the backrest of the chair.
- Place the belt around patient's waist and close the belt through the velcro strap.
- Grasp the patient and the backrest of the chair with the abdominal strap.
- Close the buckle, adjust the length of the strap while being careful not to compress the abdomen of the patient.
- Close the buckles of the shoulder belt and adjust their length (see front page).

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.