



This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

Y & PELVIC SLIM BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL




**TO PREVENT FORWARD
SLIDING AND MAINTAIN
THE CHEST IN UPRIGHT
POSITION FOR ALL CHAIRS
EXCEPT SHELLS**



Maximum useful area: + 230 cm
Belt height and width
pelvic part: + 11 cm
Width strap: + 48 mm
Buckle: 50 mm

Its shoulder heights are separately adjustable to help position and compensate for a bad position.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm (the backrest and seat of the chair must have a separation space).
- Waist (in cm)

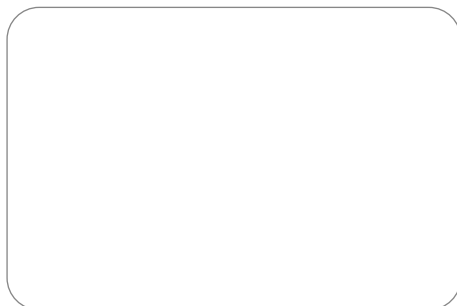

CSYP2HP-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



Y & PELVIC SLIM BELT

HOSPITAL RANGE

Combination of polyurethan foam
and on both faces
Jersey/PU coating
100% polyester bias
100% polyester strap
Nylon buckle
Thermo compressed foam

INSTRUCTION FOR USE

- Position the pelvic part on the seat of the chair and let the buckle go out towards the back of the chair.
- Let the patient sit on the chair.
- Position the belt around the patient's waist.
- Close the belt using the velcro strap.
- Place the chest belt being careful that the shoulders pads are positioned on the patient's shoulders.
- Pass the strap of the back part of the belt behind the backrest of the chair.
- Before closing it and without adjusting it, check that the back strap has a female buckle looking downwards and 2 female buckles looking upwards.
- Close the 4 rear buckles, adjust

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.