



*This medical device is intended for patient positioning.  
This is not a restraint system.*

*Before use, it is important to check the status of the device: stitching, state of the buckles, slings,  
fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## ARM SLINGS

**SUPPORTS AN ARM OR A FRONT FRACTURED ARM**



**THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL**




### ARM SLING

Support of a plastered arm, arm support at an adjustable angle with thumb rest, reversible right and left.

They are comfortable and equipped with a thumb rest.  
The addition of a back strap keeps the sling against the body of the user (ECBSM).

### ARM SLING WITH DORSAL STRAP

Complete immobilization of the arm with thumb rest, reversible right and left.

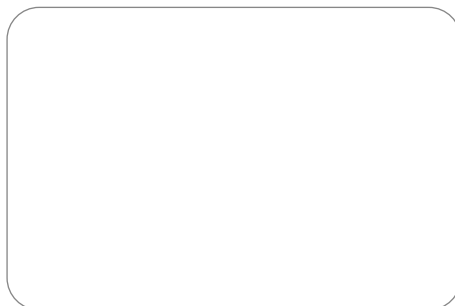

ECB-PF / ECBSM-PF

v 05.12/2020

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



# ARM SLINGS

100% polyester mesh  
100% cotton bias  
Cotton straps, polyester quilting  
Nylon buckles

## INSTRUCTIONS FOR USE to use only for the indicated use: SUPPORTS AN ARM OR A FRONTED FRONT ARM

• After placing the arm in the sling, pass the strap 1 around the neck and adjust it according to the desired position of the arm.

• For the model with back strap allowing a «bend to the body» immobilization, pass the strap 2 around the waist and fasten it with the quick buckle.



v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.