



Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

UNIVERSAL ECO HAMMOCK SLING



Compatible with
boned headrest
(TAMOV).




**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



**NOT SUITABLE FOR DEMENTED
AND SPASTIC PERSONS**

- CE certified and manufactured in France, the Universal Eco Hammock is recommended for "lifting from the floor" transfers in a lying position.
- Its simple and indented cut on the sides makes it the preferred sling to let it under the patient in a chair as it shows no thickness and folds which cause decubitus ulcers and discomfort.
- The installation must be done in bed in a lying position at 180° by practicing a lateralization.
- Moreover, it is preferred for the bath or shower, because it is made with a high tenacity polyester net in order to evacuate the water.

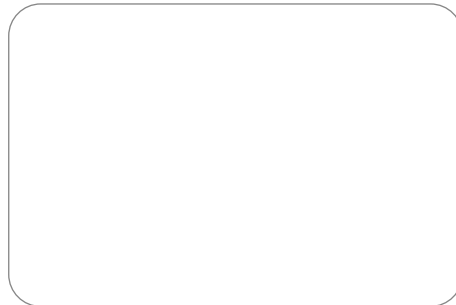

HA-RBA-PF-TU

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




UNIVERSAL ECO HAMMOCK SLING

SIZE & DIMENSIONS (in cm)

	A	B	C
One size fits	50	61	92

Composition
100% polyester fabric
40mm polyester strap - 1450 DaN
20mm polyester bias

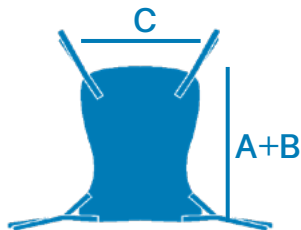

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

- Lateralize the patient. Fold the hammock in two along the vertical axis.
- Place the fold along the vertical axis at the center of the patient's back.
- Place half of the sling on the free side of the patient.
- Fold the side of the sling on the side in support of the patient.
- Turn the patient on his back with half the sling resting on the free side.
- Lateralize again the patient on the previously free side of the sling.
- Release the previously folded part and spread it out.
- Place the patient on his back on the sling deployed.
- Hang the high and low hangers on the patient lift's spreader bar.

SETTINGS

The attachment system, with multiple hangers, allows to vary the position of the person.

CAUTION

Read the instruction manual before using the slings.

The backrest hangers must be hung on the same color.

Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.

Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.