



*Prior to each use, it is important to check the status of the device:  
seams, state of the straps and of the fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## FASTENERS



### PADDED WRIST

**LIGHT WRISTS SUPPORT FOR AGITATED  
PATIENT**




### SECURITY WRIST FASTENERS

**FOR A STRONGER CONTENTION FOR VERY  
AGITATED PATIENTS**



### ANKLE FASTENERS

**MAINTAINS THE PATIENT'S  
ANKLES, WHILE ALLOWING  
FREEDOM OF MOVEMENT.**

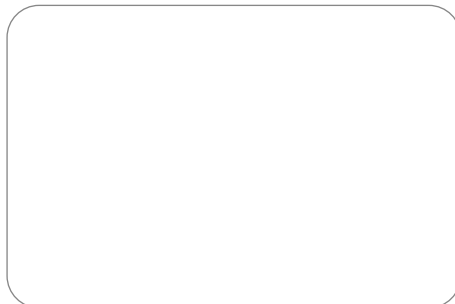

**THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL**



LMATPO-PF / LMATPOS-PF / LMATCH-PF

v 05.12/2020

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



# FASTENERS

### Composition :

**PADDED WRIST FASTENER**  
Combination of polyurethan foam and jersey/cotton polyester fabric  
100 % polyester fur  
100 % polyester velcro strap  
100 % polyester strap  
Nylon buckle

### Composition :

**SECURITY WRIST FASTENER**  
Combination of polyurethan foam and jersey/cotton polyester fabric  
100 % polyester velcro strap  
100 % polyester strap

### Composition :

**ANKLE FASTENER**  
Combination of polyurethan foam and jersey/cotton polyester fabric  
100 % polyester fur  
100 % polyester velcro strap  
100 % polyester strap  
Nylon buckle

## INSTRUCTIONS FOR USE

### TO BE USED ONLY FOR THE INDICATED USE

#### PADDED WRIST FASTENER

#### **LIGHT WRISTS SUPPORT FOR AGITATED PATIENT**

Set the wrist, on the fur side, in the bracelet.

With the help of the velcro, close the bracelet.

Wrap the strap around the object serving for fastening the fastener wrist.

Adjust the length of the belt with the nylon loop.

#### SECURITY WRIST FASTENER

#### **FOR A STRONGER CONTENTION FOR VERY AGITATED PATIENTS.**

Route the strap located at the sharp end of the bracelet into the slot (1).

Get the wrist of the person through this loop (2).

Fit to immobilize the wrist and close the bracelet using the Velcro, then move the strap through the loop (2).

Tie the straps around the object serving as fixing the security wrist fastener.



#### ANKLE FASTENER

#### **MAINTAINS THE PATIENT'S ANKLES, WHILE ALLOWING FREEDOM OF MOVEMENT.**

Set each ankle, on the fur side, in each bracelet.

Using the velcro, close each bracelet.

Route the strap around the object serving for fastening the ankle fastener.

Adjust the length of the belt with the nylon loop.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.