



This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

SECURITY MITTS



integrated
aluminium
stiffener

USES :
**COMFORTABLE, THEY
PREVENT THE RESTLESS
PATIENT FROM TEARING
HIS CARE DEVICES AS
WELL AS HIS INCONTI-
NENCE PROTECTIONS.**




**THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL**



Description

The breathable net part on the top increases comfort and the integrated aluminium stiffener prevents the hand flexion.

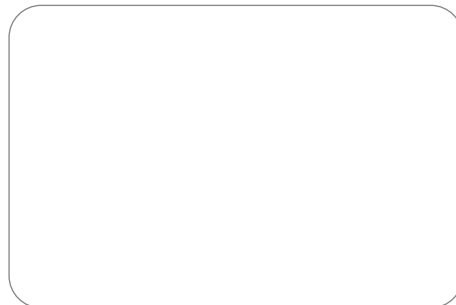

LMMFL-HP-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



SECURITY MITTS

HOSPITAL RANGE

Combination of polyurethan foam
and on both faces jersey/PU coating

100% polyester net

100% polyester bias

100% polyester strap

Nylon buckle

Aluminium stiffener

INSTRUCTIONS FOR USE

COMFORTABLE, THEY PREVENT THE RESTLESS PATIENT FROM TEARING HIS CARE DEVICES AS WELL AS HIS INCONTINENCE PROTECTIONS.

- Place the patient's hand in one of the security mitt, net part on the upper part of the hand.
- Close the nylon buckle, then adjust the tightening.
- Secure it with the velcro strap.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.