

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

HAMMOCK WITHOUT CROSSING SLING (WITHOUT HEADREST) BATH MODEL



Compatible with
the removable headrest
(TAMOV).




**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



• CE certified and manufactured in France, the Bath Hammock Without Crossing is recommended for bed to chair and from chair to bed transfers for amputees and for "lifting from the floor" transfers in a lying position with the addition of headrest.

• It is a hammock sling with a U-shape which avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.

• Moreover, this sling avoids hips tightness and the foam leggings reduce the pressures exercised on the thighs.

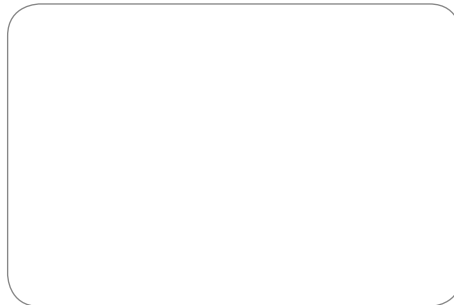

SUCBAHA-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient


These products must not be in direct contact with a wounded skin.

HAMMOCK WITHOUT CROSSING SLING (WITHOUT HEADREST) BATH MODEL

SIZES & DIMENSIONS (in cm)

	A	B	C
S	41	53	75
M	46	57	83
L	50	61	92
XL	55	66	102

Composition
Jersey 100 % polyester
Sangle polyester 40 mm - 1450 DaN
Mousse PE 50 kg/m³
Biais polyester 40 mm

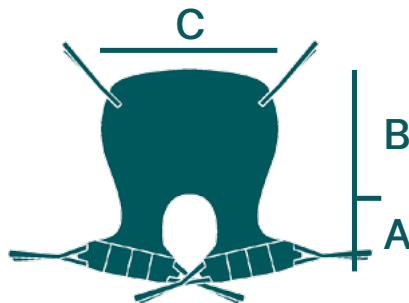

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

Place the patient in recovery position. Fold the hammock in 2 part by respecting the vertical axis.

- Position the fold according the vertical axis in the middle of the patient's back.
- Put the half sling on the patient's free side.
- Fold the extreme side of the sling supporting by the patient
- Reverse the patient on his back with the half sling places on the free side
- Recover the patient on the sling on side previously available
- Release the part previously folded and spread it
- Place the patient on his back on the sling deployed
- Hook the top lines and the bottom lines in the middle of the patient hoist

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 02.03/2019



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**