

*Prior to each use, it is important to check the status of the device:  
seams, state of the straps and of the fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## PELVIC SUPPORT BELT

FOR ALL CHAIRS

Complies with the ANSM's recommendation concerning awareness of the risk of suffocation when securing a patient using a bed rest device.




**THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL**



USE :  
**PREVENT PATIENTS FROM  
FALLING, TO COMPENSATE  
SLACKENING OF ABDOM-  
INAL MUSCLES AND TO  
POSITION IN BED.**

The Bed Support Belt is designed to prevent falls in bed.  
It gives the patient a secure positioning.  
Compatible with the NAUSI-SURE system (magnetic closure).

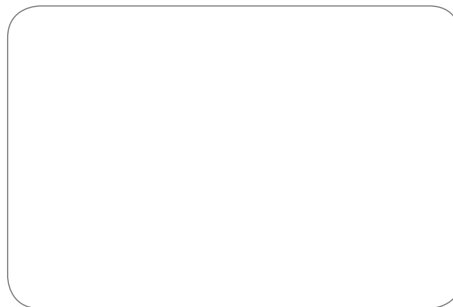

CMLPHP-PF

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

# PELVIC SUPPORT BELT

Complies with the ANSM's recommendation concerning awareness of the risk of suffocation when securing a patient using a bed rest device.

**HOSPITAL RANGE**  
Combination of polyurethan foam  
and jersey/cotton polyester fabric  
100 % polyester jersey  
100 % polyester bias  
100 % polyester strap  
Nylon buckle

## INSTRUCTIONS FOR USE

to use only for the indicated use:

### PREVENTING FALLS, SUPPORTING ABDOMINAL RELEASE AND POSITION IN THE BED

- Put the waistband around the waist of the person, the slit on the top (photo 1).
- Pass the strap of the narrow end into the slot of the belt.



Photo 1

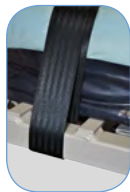


Photo 2



Photo 3

- Pass the two straps of the belt part around the fixed amounts of the bed base (use bed-photo 2) or behind the back of the chair.
- Connect the 2 straps of the belt by passing them on the belly of the patient with the click-clack loop and adjust the tension (use bed). This strap can be secured with the NAUSI-SURE device (photo 3).

- Pass the first leg under the thigh on the same side and pass the strap in the loop on the same side.

- Replicate the operation for the other leg.

- Pass each leg strap around the fixed post of the bed base and connect them with the click-clack loop by passing them over the patient (use bed-photo 4-5) or behind the back (use chair). Adjust the voltage.



Photo 4



Photo 5

## BED USE IMPORTANT:

- **MAKE SURE THAT THE POSITIONING OF THE DEVICE DOES NOT INTERFERE WITH THE USE OF BUSTING (RISK OF COMPRESSION).**
- **WELL POSITION THE DEVICE AT THE FIXED PART OF THE ARTICULATED BED BASE.**
- **CLOSE THE BARRIERS AND PUT THE BED IN THE LOWER POSITION.**

v 02.03/2019



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.