



Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

STANDARD HAMMOCK SLINGS



Compatible with
removable headrest
(TAMOV).




**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



**NOT SUITABLE FOR DEMENTED
AND SPASTIC PERSONS**



- CE certified and manufactured in France, the Standard Comfort Hammock is recommended for bed to chair to bed transfers where the sling remains on the chair.
- Its simple and indented cut on the sides makes it the preferred sling to remain under a patient in a chair, as it shows no thickness and folds that cause decubitus ulcers and discomfort.
- The leggings comfort foam reduces the shearing effect and the installation must be done in bed in a lying position at 180° by practicing a lateralization.
- Moreover, this sling avoids hips tightness.

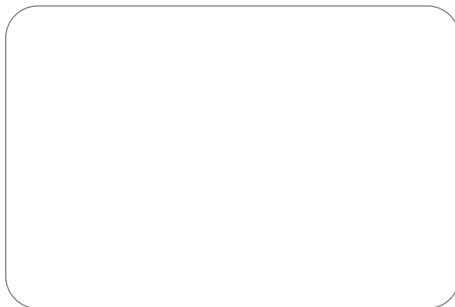

SHA2-PF / SHAC2-PF / SHA2BA-PF / SHAC2BA-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




STANDARD HAMMOCK SLINGS

SIZES & DIMENSIONS (in cm)

	A	B	C
S	57	62	54
M	61	67	65
L	65	69	76
XL	70	71	86

Composition
100% polyester fabric
40mm polyester strap 1450 DaN

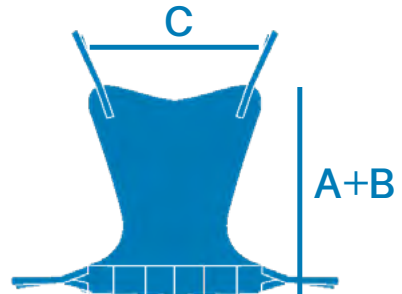

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

This sling has a 4 points hanging system to the spreader bar :

- 2 on the top of the sling (at the shoulders' level)
- 2 on the bottom of the sling (at thigh level)

Position the lower part of the back at the coccyx so that the loops of the hanging straps are on both sides of the patient.

If need be, position the lower part of the head pad under the patient's shoulders (the top of the head pad positioned on the top of the head). The head pad will be maintained by the shoulders pressure.

Hang each hanging straps on the spreader bar without crossing.

ADJUSTMENTS :

The multiple hanging straps allow various positions : sitting, semi-sitting or lying position.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**