



*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

FAST ADJUSTABLE SLINGS




Compatible with
removable headrest
(TAMOV).



**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



- CE certified and manufactured in France, the Fast Adjustable Comfort Sling is recommended for bed to chair to bed transfers requiring a management of the head support.
- We recommend using it for bed to chair to bed transfers requiring a management of the head support (with the addition of removable headrest).

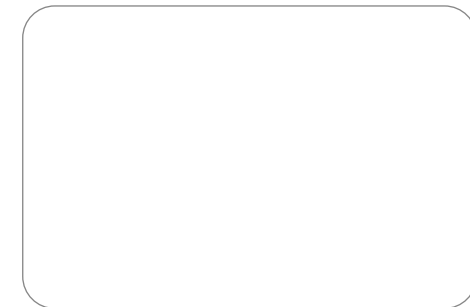

SST-PF / SML-PF / SSTBA-PF / SMLBA-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




FAST ADJUSTABLE SLINGS

SIZES & DIMENSIONS (in cm)

	A	B	C
S	57	62	54
M	61	67	65
L	65	69	76
XL	70	71	86

Composition
100% Polyester Fabric
150Kg/M3 PE foam
Sling 100% Polyester

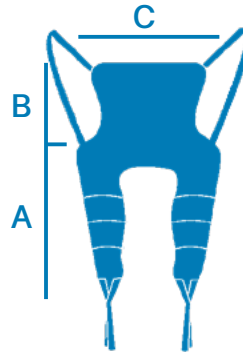

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.