


Conceived to be used with the EASYLEV 5 range (page 18), the Standard Sling is available in 4 different sizes.

Placed under the armpits, its full padding under the arms distributes traction on the patient thus avoiding painful pressure and supports the chest area.

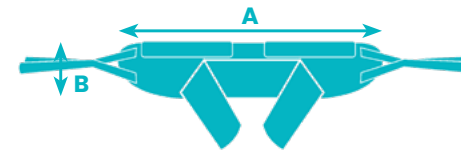

<b>S</b>	A	81	<b>L</b>	A	107
	B	21		B	29
<b>M</b>	A	94	<b>XL</b>	A	123
	B	25		B	33



REFERENCES	
Jersey	SAPVR-PF-S
Jersey	SAPVR-PF-M
Jersey	SAPVR-PF-L
Jersey	SAPVR-PF-XL



Jersey

## TRANSFER (sitting position)



1

Position the "under thighs" strap and close it adjusting it without tightening too much.



2

Handlebar in lower position: Position the **yellow loops** on the central hooks of the handlebar.



3

Operate the actuator until halfway.