



Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

QUICK TOILET SLING




Compatible with
removable headrest
(TABAL).



**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



- CE certified and manufactured in France, the Quick Toilet Sling is recommended for toilet and daily change transfers.
- Its extended U-shape completely frees the patient's buttocks thus facilitating the positioning to the toilet, and improving the installation and removal of the sling.
- The leggings and back part in foam distribute the pressures exercised on the thighs.
- This sling is not recommended for patients with very low tone.

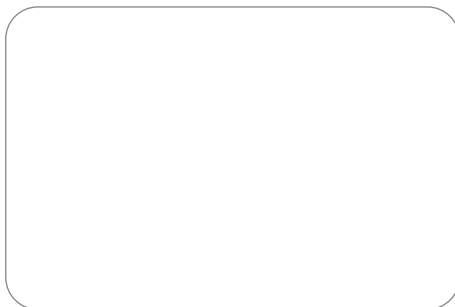

STO-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




QUICK TOILET SLING

SIZES & DIMENSIONS (in cm)

	Waist size	A	B	C
S	70/90	93	25	81
M	90/115	105	25	95
L	115/140	115	25	109
XL	140/165	125	25	123

Composition
100% Polyester Fabric
50Kg/M3 PE foam
100% Polyester
20Kg/M3 35mm PE foam

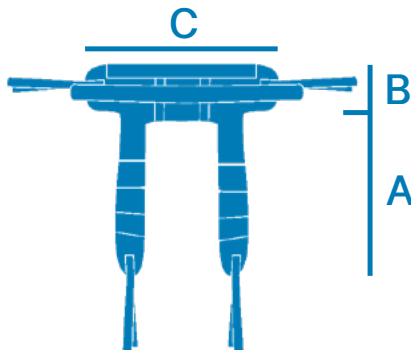

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lowest part of the back on the shoulder blades, so that the hanging loops are positioned on both sides of the patient. Place the hanging loops under the patient's arm, so that the foam parts are positioned under the armpits. Close the chest loop.

Slightly raise the right leg of the patient, pass from the inside of the thighs outwards the loop of the right thigh support, do the same for the left thigh support. Hang both back loops and both thigh supports to the spreader bar.

ADJUSTMENTS :

The different loops on the hanging straps allow various sitting positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL OR THE USER MUST BE EDUCATED TO SET IT UP.