



Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

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
ECO SLING



THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL



- CE certified and manufactured in France, the Eco Sling is available in one size fits and is used with all stand-up lifts equipped with a hook system.
- It is placed under the armpits or on the lumbar based on the device used.
- Its padding under the arms distributes traction on the patient and avoids painful pressure, particularly under the armpits and at the dorsal level.



SXPECO-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient


These products must not be in direct contact with a wounded skin.



ECO SLING

SIZE & DIMENSIONS (in cm)

	A	B
One size fits	29	106

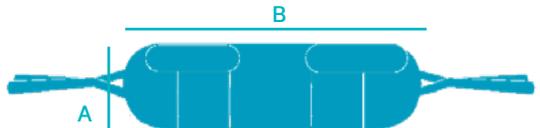

Composition
100 % polyester fabric
40 mm polyester strap - 1450 DaN
50 kg/m³ PE foam

INSTRUCTIONS FOR USE

Do not use this sling if the size is not suitable for the patient.

Position the bottom of the backrest in the lumbar region in order that the ears of the attachment system are on either side of the person.

Place the fasteners of the backrest under the arms of the person.

Close the plastron.

Hook the two fasteners of the backrest to the stand-up lift hooks.

SETTINGS

The fastening system, with multiple hangers, allows to adjust the position of the person.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**