

Size Chart

Weight / Sizes	1m50	1m55	1m60	1m65	1m70	1m75	1m80	1m85	1m90	+ 1m90
40 kg	S	S	S	S	S	S	M	M	M	M
45 kg	S	S	S	S	S	S	M	M	M	M
50 kg	S	S	S	S	S	S	M	M	M	M
55 kg	S	S	S	S	S	S	M	M	M	M
60 kg	M	M	M	S	S	S	M	M	M	M
65 kg	M	M	M	S	S	S	M	M	M	M
70 kg	M	M	M	M	M	M	M	M	M	M
75 kg	L	L	M	M	M	M	M	M	M	M
80 kg	L	L	L	L	M	M	M	M	M	M
85 kg	L	L	L	L	M	M	M	M	M	M
90 kg	L	L	L	L	L	L	M	M	M	M
95 kg	XL	XL	XL	XL	L	L	M	M	M	M
100 kg	XL	XL	XL	XL	L	L	L	L	L	L
110 kg	XL	XL	XL	XL	L	L	L	L	L	L
120 kg	XL	XL	XL	XL	XL	XL	L	L	L	L
130 kg	XL	XL	XL	XL	XL	XL	XL	XL	XL	XL
160 kg	2 XL	2 XL	2 XL	2 XL	2 XL	2 XL	XL	XL	XL	XL
190 kg	3 XL	3 XL	3 XL	3 XL	2 XL	2 XL	XL	XL	XL	XL
220 kg	3 XL	3 XL	3 XL	3 XL	3 XL	2 XL	2 XL	2 XL	XL	XL
250 kg	4 XL	4 XL	3 XL	3 XL	3 XL	3 XL	3 XL	2 XL	2 XL	2 XL
280 kg	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	3 XL	3 XL	3 XL	3 XL
320 kg	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL	4 XL

Headrests



TAMOV



TABAL



TAMAX

Straps Positioning



Low straps crossed between legs



Low straps crossed under legs



Low straps uncrossed between legs

PATIENT LIFTS SLINGS RANGE

User Guide

Sizing is an important element when deciding on a sling.

In addition to comfort, choosing the appropriately sized sling will improve the patient's safety during transfer.

That's why, a size colour-code of the edges has been put in place in order to quickly identify the size of each sling:

- to the size S corresponds the Yellow colour
- to the size M corresponds the Red colour
- to the size L corresponds the Green colour
- to the size XL corresponds the Blue colour
- to the size 2 XL, 3XL, 4XL corresponds Orange colour
- to the size TU (one size fits) corresponds the Black colour

In order to choose the right sling, the caregiver shall consider the 3 following factors with the patient:

- height
- waist size
- thighs size

HEADRESTS

The addition of a headrest helps to take in charge the patient's head and protects the cervical.