



*This medical device is intended for patient positioning.  
This is not a restraint system.*

*Before use, it is important to check the status of the device: stitching, state of the buckles, slings,  
fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## YZA JACKET


FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL



A vest, plus a single size pelvic module, allow you to go from a bust fit to an integral fit according to changing needs without size constraints.

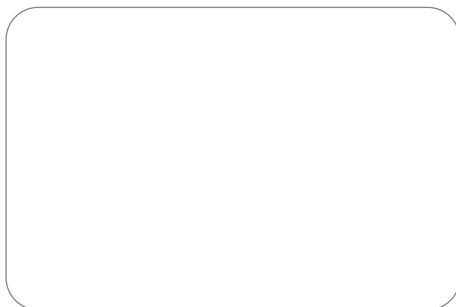

YZA2GBHP-PF

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient

- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



# YZA JACKET

## HOSPITAL RANGE

Polyester-coated polyester jersey

lined with polyurethane foam

Bias 91% polyamide - 9% elastane

100% polyester webbing

PVC Shoulder Pad

Nylon buckle

## INSTRUCTIONS FOR USE

**DO NOT USE THIS STRAP IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.**

The YZA Jacket comes in 2 modular variants:

### 1-YZA Bust Jacket (YZA-GB)

a - The pelvic part must be returned inside the vest

b - Put on the bust vest, positioning the shoulder pads well on the shoulders of the person, the buckles behind the back of the chair.

c - Position the belt at the waist of the person.

d - Gird the patient and the back of the chair with the lap strap.

e - Close the loop and adjust the length, taking care not to compress the abdomen.

f - Pass a shoulder strap under the abdominal support strap, connect the loops, and adjust its length.

### 2 -The YZA Integral Jacket (YZA-GI) consists of the YZA Bust Jacket to which is added a Pelvic Support Module.

a - Position the pelvic support module on the chair seat, and secure the 25 mm strap to the back of the chair with minimum play.

b - Sit the patient on the chair and on the strap.

c - Put in place the bust vest, positioning the shoulder pads well on the shoulders of the person, the loops behind the back of the chair.

d - Pass the straps behind the back of the chair.

e - Close the nylon loop of the lap strap, then adjust the length of the strap, being careful not to compress the patient's abdomen.

f - Pass the strap of the pelvic support in the passer of the pelvic part. Close the 50 mm loop of the pelvic support by adjusting its length.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.