



This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygiene reasons and respect for sanitary protocol, textile articles that have been opened will not be returned or exchanged.


YZA BELT

FOR ALL CHAIRS



Its shoulder heights are separately adjustable to help position and compensate for a bad position.

- Available in 4 sizes S, M, L, XL
- Maximum useful area: 230 cm (the backrest and seat of the chair must have a separation space).
- Waist (in cm)

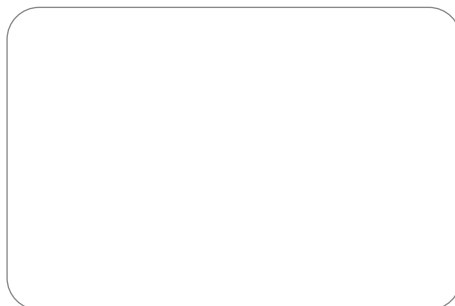

**THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL**



YZA2CAHP-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



YZA BELT

HOSPITAL RANGE

Lined polyurethane coated polyester
jersey polyurethane foam
Bias binding 91% polyamide - 9% elastane
100% polyester webbing
PVC shoulder pad
Nylon buckle

INSTRUCTION FOR USE

The YZA Belt is declined under 4 modular versions :

1. the Abdominal YZA Belt.
2. the Bust YZA Belt is composed of the Abdominal YZA Belt to which is added a bust modular element.
3. the Pelvic YZA Belt is composed of the Abdominal YZA Belt to which is added a pelvic modular element.
4. the Integral YZA Belt is composed of the Abdominal YZA Belt to which is added a bust modular element and a pelvic modular element.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.