

# LIFTING FROM FLOOR

## What type of slings?

NAUSICAA Medical provides 6 types of slings to be able to perform a transfer from the floor:

- Universal Eco Hammock Sling (page 65)
- U-Shaped Sling (page 66)
- Hammock Without Crossing Sling (page 71)
- Enveloping Sling (page 72)
- Ergonomic Sling (page 72)
- Backrest Sling (page 73)

## How to do it?

When collecting from floor, you should:

1. Secure the patient's neck using a rigid headrest (removable or sewed):
  - If the sling does not have a headrest, you must accompany the patient's neck.
2. The patient's head must be opposite the mast avoiding thus shocks during lifting.
3. Do not forget to open the patient lifter's legs to facilitate the passage (one along the body and the second under the bend of both knees).

