



This medical device is intended for patient positioning.

This is not a restraint system.

Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.

INTEGRAL JACKET

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL




**TO MAINTAIN THE CHEST
IN UPRIGHT POSITION
FOR ALL CHAIRS**

Maximum useful scope: + 220 cm
Buckle: 40 & 25 mm

Made in a breathable fabric, it is easy to put in place thanks to its unbreakable shoulders.

- Declined in 3 sizes S, M, L corresponding to the sizes of clothing.
- Maximum useful area: 220 cm

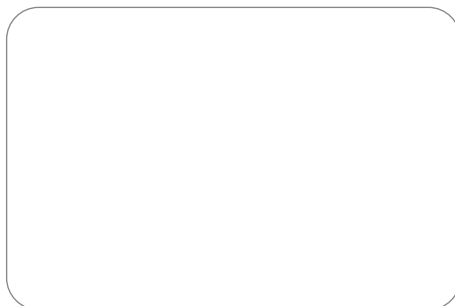

CGI3-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



INTEGRAL JACKET

HEMOCARE RANGE
Bi-laminate warp and weft Polyester
fabric with Polyurethane membrane
100% cotton bias
100% polyester strap
Nylon buckle

INSTRUCTIONS FOR USE

- Open the buckles situated on the shoulders.
- Place the complete jacket.
- Close the buckles.
- Cross the jacket on the front.
- Pass the belt strap through the provided slit.
- Grasp the backrest of the chair while being careful not to compress the abdomen of the patient.
- Pass the end of the additional delivered strap in the back loop.
- Close the 40mm buckle.
- Adjust the length of the strap through the 25mm nylon buckle to avoid that the person falls forward.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.