



*This medical device is intended for patient positioning. This is not a restraint system. Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.*

## PELVIC COMFORT BELT

FOR ALL CHAIRS



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL




**TO PREVENT FORWARD  
SLIDING FOR ALL CHAIRS  
EXCEPT SHELLS**

Périmètre maximum utile :  $\pm$  230 cm  
Boucle : 50 mm

Its internal STERITHANE protection makes it waterproof while being breathable.

- Available in 3 sizes S, M, L
- Maximum useful area: 230 cm
- Waist (in cm)

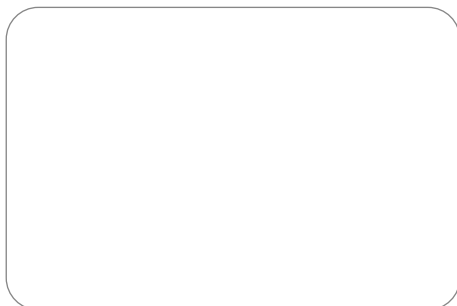

CPCF3HP-PF

### Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



## PELVIC COMFORT BELT

### HOSPITAL RANGE

Combination of polyurethan foam and on both faces jersey/PU coating.

100% polyester bias  
100% polyester strap  
nylon buckle

### INSTRUCTIONS FOR USE

- Place the part « pelvic holder » on the seat of the chair, the narrowest part to the backrest.
- Pass the strap of the narrowest part around the backrest of the chair, firmly adjust it and close the buckle.
- Sit the patient on the chair and raise the belt part between his legs and adjust the belt around the patient's abdomen.
- Adjust the pelvic part to avoid creases.
- Pass the strap which has not been used yet behind the backrest, close the buckle, adjust the length of the strap while being careful not to compress the abdomen of the patient.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.