



Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.

SIDE RAIL PADS



PRO-BAR-M



PRO-BAR-L



**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**




FLAME RETARDANT COVERS



removable cover

The foam protectors are installed on the bed rails thus preventing the patient from bumping and passing the limbs through the barriers which protects him from contact with the metal. The quick and easy attachment to the bed rails is done with quick buckles (25 mm). The removable protective cover is classified M1 fire.

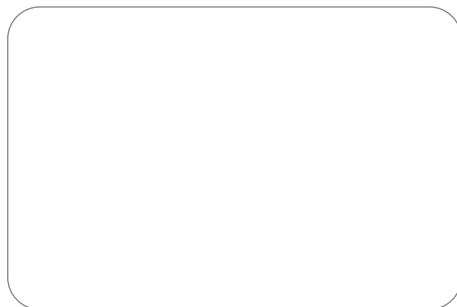

PROBAR-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



SIDE RAIL PADS

Composition:
Flame retardant M1 PVC/PU coating
PU foam
Polyester strap
Nylon buckle

The Siderail Pads are installed on the bed barriers and prevent the patient from bumping and moving its limbs through the bed rails, thus protecting him from contact with the metal.

The quick and easy fixations to the siderails is made thanks to adjustable buckles (25 mm).

The removable foam cover is manufactured in a flame retardant fabric (classified CRIB 5).

INSTRUCTIONS FOR USE :

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

Place the cotsides bumpers inside the side rails, zipper down, against the mattress.

- Close the buckles and adjust tight the straps around the tubes of the side rails.
- Check if the cotsides bumpers are well fixed to the side rails, without any possibility to turn around.

CLEANING :

- Clean with a soft cloth and warm soapy water on the surface of the cotsides bumper.
- Desinfect the surface of the cotside bumper with any used desinfection product.

Cleaning of the covers :

- Never wash the covers together with the foam (see front page).

v 05.12/2020



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**