

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*


WALKING AID SLING



**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



- CE certified and manufactured in France, the Walking Aid Sling adapts to the morphology of each patient by its multitude of sizes (S, M, L, XL) and is used with all stand-up lifts whose platform can be removed.
- It is used as a walking aid in order to improve patient's mobility and independence.

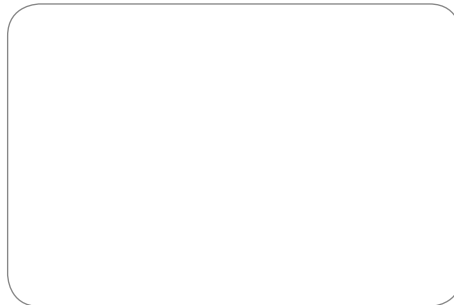

SWYDE2-PF

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :


- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

WALKING AID SLING

SIZE & DIMENSIONS (in cm)

	Plastron	A	B	C
S	70 / 90	21	78	72
M	90 / 115	25	92	76
L	115 / 140	29	106	80
XL	140 / 165	33	120	84

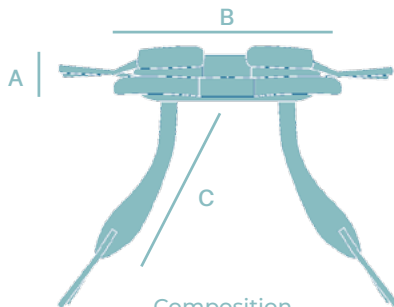

Composition

100 % polyester fabric
40 mm polyester strap - 1450 DaN
100 kg/m³ PE foam
20/25 kg/m³ PE foam
20 mm polyester bias

CAUTION

Read the instruction manual before using the slings.

The backrest hangers must be hung on the same color.

Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.

Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.

v 02.03/2019



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.