



*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

BACKREST SLING



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
**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



• CE certified and manufactured in France, the Backrest Sling is recommended for bed to chair to bed transfers and for "lifting from the floor" transfers in a lying position for specific pathologies.

• Its U-shape associated with the net portion beneath the buttocks make the installation and removal comfortable for the patient and accommodating for the caregiver.

• Its back part associated with the headrest in rigid foam and the full stiffener allow to protect the spine during transfers.



SGLDOS-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




BACKREST SLING

SIZES & DIMENSIONS (in cm)

	A	B	C
S	59	88	75
M	66	92	83
L	71	96	92
XL	77	101	102

Composition

100% Polyester fabric
High tenacity 100% polyester net
100 % Polyester strap
50kg/m³ PE foam – 100kg/m³ PE foam
20mm Polyester bias
Aluminium stiffener 15x3mm

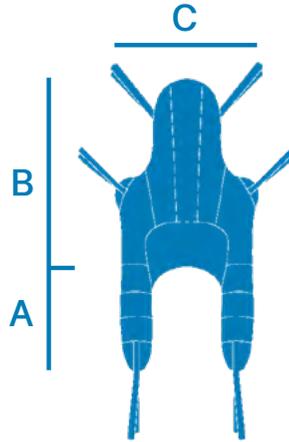

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This sling has a 6 points hanging system to the spreader bar :

- 2 on both sides of the top of the back
- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lower part of the back at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Do not cross the hanging straps. Hang both hanging straps located at the top of the back and the ones located on the back as well as the hanging straps of the thigh supports to the spreader bar.

ADJUSTMENTS :

The multiple hanging straps allow various sitting positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**