



*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*


ENVELOPING SLING



**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



- CE certified and manufactured in France, the Enveloping Sling is recommended for bed to chair transfers and for "lifting from the floor" transfers in a lying position.
- The integrated and boned headrest in addition to the enveloping shape of the sling make it suitable for the transfers of agitated and anxious patients.
- Its U-shape avoids the placement of the sling under the patient's buttocks, making the installation and removal comfortable for the patient and accommodating for the caregiver.

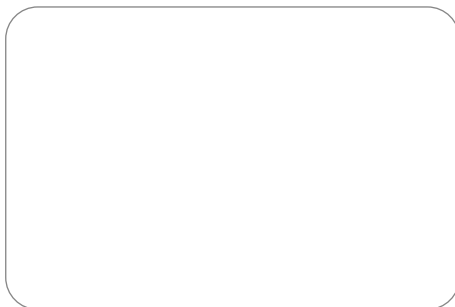

SLK-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




ENVELOPING SLING

SIZES & DIMENSIONS (in cm)

	A	B	C
S	59	88	108
M	66	92	120
L	71	96	134
XL	77	101	150

Composition
100% polyester fabric
40mm polyester strap - 1450 DaN
50kg/m³ Polyester foam
Aluminium stiffener 15x3mm

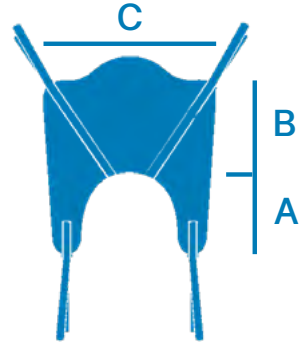

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

The widest part of the sling constitutes the back of this type of sling. The branches of the U constitute the thigh support.

This type of sling has a 4 points hanging system to the spreader bar :

- 2 on both sides of the back
- 1 at each end of the thigh support.

Position the lower part of the sling at the coccyx so that the loops of the hanging straps are on both sides of the patient. Slightly raise the right leg of the patient, pass the right thigh support under the right thigh. Do the same for the left leg. Both hanging straps are positioned between the legs of the patient. Cross these hanging straps. Hang both hanging straps of the back and hanging straps of the thigh supports to the spreader bar.

ADJUSTMENTS :

The different loops on the hanging straps allow various sitting positions.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 04.07/2020



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**