



*Prior to each use, it is important to check the status of the device:  
seams, state of the straps and of the fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*


## ERGONOMIC SLING



**THIS MEDICAL DEVICE  
MUST BE IMPLEMENTED  
BY A TRAINED HEALTH  
PROFESSIONAL**



- CE certified and manufactured in France, the Ergonomic Sling is recommended for bed / chair / bed transfers and floor pickups when lying down.
- Its U-shaped shape avoids the placement of the strap under the patient's buttocks, making the placement and removal more pleasant for the patient and easier for the caregiver.
- Its back section, combined with the rigid foam headrest, lined with a 3D fabric, increases comfort and protects the spine during transfers.

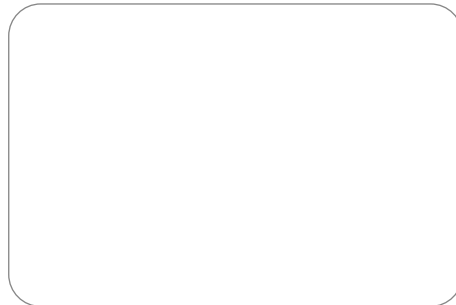

SMB-PF

v 05.12/2020

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.




# ERGONOMIC SLING

## SIZES & DIMENSIONS (in cm)

	A	B	C
S	59	88	75
M	65	92	83
L	71	96	92
XL	77	101	102

### Composition

- 100% polyester jersey
- 3-dimensional polyester fabric
- Polyester webbing 40 mm - 1450 DaN
- PE foam 50 kg / m<sup>3</sup>
- PE foam 100 kg / m<sup>3</sup>
- 40 mm polyester straps
- Polyester bias 20 mm

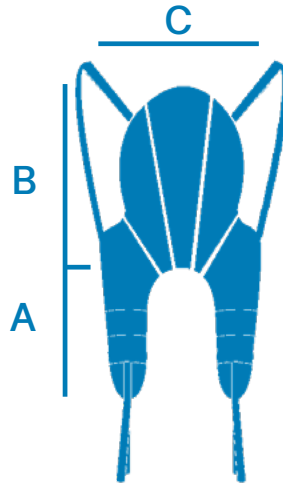

## INSTRUCTIONS FOR USE

**DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.**

The backrest of this type of strap, is constituted by the wide part of the strap.

The thigh supports, meanwhile, are constituted by the branches of the U.

This type of strap has a system of attachment to the flail of the lifter in four points:

- 2 on both sides of the backplash,
- 1 at each end of the thigh supports.

Position the bottom of the backrest at the level of the coccyx so that the lines of the attachment system are on both sides of the person. While lifting the person's right leg slightly, move the right thigh support under the right thigh. Proceed in the same way for the left leg. The two leg support systems are located between the legs of the person. Pass these two systems through each other. Hang the two backrest fasteners and the two thigh support attachments to the flail of the mobile lift.

## SETTINGS

The hanging system, multiple lines, allows to vary the position of the person.

## CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.**