



*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*


FAST ADJUSTABLE ECO SLING (WITHOUT HEADREST)



**THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL**



The Fast Adjustable Eco Sling (without headrest) is recommended for bed to chair to bed transfers requiring a management of the head support.
The foam leggings reduce the pressures exercised on the thighs and the hooks with two settings (especially suitable for 2-point spreader bars) allow fast combinations for sitting, semi-sitting and lying transfers.

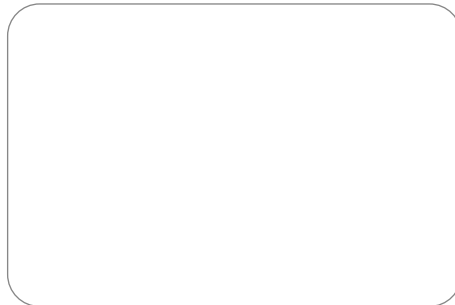

SSTRP-PF-TU

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient


These products must not be in direct contact with a wounded skin.

FAST ADJUSTABLE ECO SLING (WITHOUT HEADREST)

SIZE & DIMENSIONS (in cm)

	A	B	C
One size fits	65	69	76

Composition
Jersey 100 % polyester
Sangle polyester 40 mm - 1450 DaN
Mousse PE 50 kg/m³

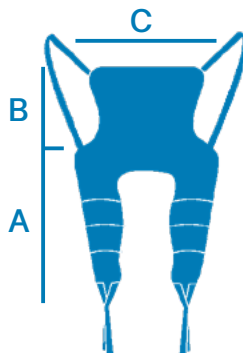

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 03.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.