



*Prior to each use, it is important to check the status of the device:  
seams, state of the straps and of the fabric.  
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*

## UNIVERSAL ECO TOILET SLING




Compatible with  
boned headrest  
(TABAL).



**THIS MEDICAL DEVICE  
MUST BE IMPLEMENTED  
BY A TRAINED HEALTH  
PROFESSIONAL**



The Universal Eco Toilet Sling is recommended for toilet and daily change transfers. Its extended U-shape completely frees the patient's buttocks thus facilitating the positioning to the toilet, and improving the installation and removal of the sling.

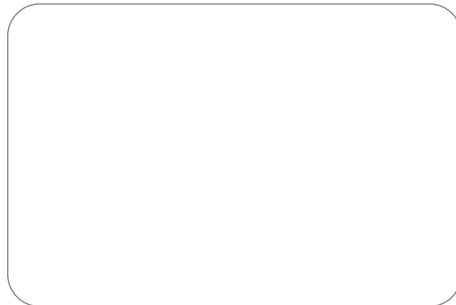

STORP-PF-TU

## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



# UNIVERSAL ECO TOILET SLING

## SIZE & DIMENSIONS (in cm)


	A	B	C
One size fits	75	25	108

### Composition

Jersey 100 % polyester

Sangle polyester 40 mm - 1450 DaN

Mousse PE 50 kg/m<sup>3</sup>

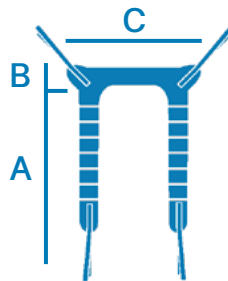

## INSTRUCTIONS FOR USE

**DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.**

The backrest of this type of strap, is constituted by the wide part of the strap.

The thigh supports, meanwhile, are constituted by the branches of the U.

This type of strap has a hanging system to the flail of the lift in four points:

1 at each end of the thigh supports.

Position the bottom of the backrest at the level of the shoulder blades so that the ears of the system of attachment are on both sides of the person. Pass the fasteners of the back splash under the arms of the person, so that the bead is under the armpits. Close the plastron.

While slightly lifting the right leg of the person, pass from the outside of the thigh inward the attachment of the right thigh support, proceed similarly for the support of left thigh. Hook the two backrest fasteners and the two thigh support clips to the flail of the lift.

2, on both sides of the backsplash

## SETTINGS

The attachment system, by multiple ears, allows to vary the sitting position of the person.

## CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL OR THE USER MUST BE EDUCATED TO SET IT UP.