

PRODUCT SHEET NAUSIFLOW 100 AUTO QUATTRO PUMP

Specifications:

- Alternating mode
- Static mode / low pressure
- Standard pressure setting
- Self management of seating positions
- 150 kg weight capacity
- Class 1 medical device
- Environmental conditions of use and storage:
 - Temperature: 5°C - 45°C
 - Humidity: 15% - 60%

Electrical Classification:

- Class II Type B, double insulation with or without ground wire.
- IPX0, do not immerse the pump in a liquid, do not spray liquid directly on the pump.
- This pump is not protected AP / APG (AP = flammable anesthetics in the air), (APG = flammable anesthetics with oxygen or nitrous oxide).

Standards: EN 60601-1, EN 60601-1-2, EN 61000-3-2, EN 61000-3-3, UL-2601-1
Electromagnetic compatibility: EN 60601-1-2: 1993
Consumption: in normal use, max 12 W
Power Cable: H05VV-F3x0.75mm²

This product complies with the requirements for the safety and health of the Directive 93/42/EC concerning medical devices and the basic requirements of Directive 86/336/EC on electromagnetic compatibility

Functions:

- 20 cm air height: optimized management of an air mattress with a 20 cm height
- 13 cm air height: optimized management of an air mattress with a 13 cm height
- Program selection: choice between 4 different programs (soft, standard, firm, seated position over 2 hours)
- 30 minutes timed care position
- Palliative care position (static, low pressure)
- Low pressure indicator: when insufficient pressure is detected, the indicator lights up and the audible alarm sounds
- Alarm stop button: turns off the alarm tone
- Power switch: turns the pump on and off



Key features :

- Automatic pressure setting
- Self management of seating positions (less than 2 hours)
- Seating positions program (over 2 hours)
- Palliative care position
- Management of air mattresses height (13 or 20 cm)
- Static mode secure management (palliative care)
- Timed care position
- Alternating and automatic static modes depending on the selected program
- Visual and audible alarm: low pressure