

ISO 13485  
BUREAU VERITAS  
Certification



NAUSICAA  
Médical  
www.nausicaa-medical.eu

MADE IN FRANCE  
IN OVER WORKSHOP  
SINCE 1992



*Prior to each use, it is important to check the status of the device: seams, state of the straps and of the fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal. For hygienic reasons and respect for health protocol, textile products have been opened will not be returned nor exchanged.*


## ECO YZA BELT



THIS MEDICAL DEVICE MUST BE  
IMPLEMENTED BY A TRAINED  
HEALTH PROFESSIONAL



An abdominal belt, plus a single-sized pelvic module allow you to switch from an abdominal support to a pelvic support according to changing needs without size constraints.

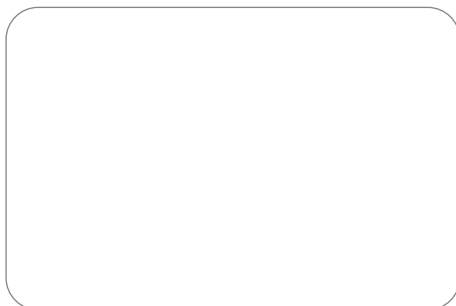

CECO3-PF

v 05.07/2020

Textile materials in  
accordance with the legislation  
**REACH**  
on undesirable substances

 NAUSICAA Médical  
12 Allée du Piot - ZA Pôle Actif  
30660 Gallargues le Montueux  
FRANCE



## Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient

- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



# ECO YZA BELT

## HOSPITAL RANGE

Polyester lined polyester jersey lined  
polyurethane foam

Bias 91% polyamide - 9% elastane

100% polyester webbing

PVC Shoulder Pad

Nylon buckle

## INSTRUCTIONS FOR USE

The ECO Belt comes in 2 modular variants:

### 1-ECO Abdominal Belt (CECO-CA)

a - Position the belt at the waist of the person.

b - Pass the straps behind the back of the chair.

c - Close the nylon loop, then adjust the length of the straps, being careful not to compress the patient's abdomen.

### 2 -The Pelvic ECO Belt (CECO-P) consists of the ECO Abdominal Belt to which is added a Pelvic Support Module.

a - Position the pelvic support on the seat of the chair, and fix the 25 mm strap to the back of the chair with the minimum of play.

b - Sit the patient on the chair and on the strap.

c - Pass the straps behind the back of the chair.

d - Close the nylon loop, then adjust the length of the strap, being careful not to compress the patient's abdomen.

e - Close the 50 mm buckle of the pelvic support by adjusting its length.

v 05.07/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL  
OR THE USER MUST BE EDUCATED TO SET IT UP.