

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*


BACKREST TOILET SLING



THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL



- It is a U-shaped hammock strap that avoids the placement of the strap under the patient's buttocks, making the placement and removal more pleasant for the patient and easier for the caregiver.
- Foam leggings reduce pressure on the thighs.
- This strap avoids the constriction of the hips.
- It acts as a pierced hammock.
- It has a headrest with attachments.

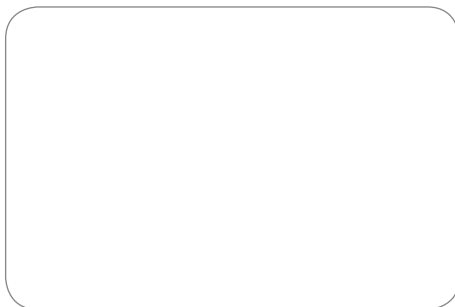

SUCPEHAT-PF-XL

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient


These products must not be in direct contact with a wounded skin.

BACKREST TOILET SLING

SIZES & DIMENSIONS (in cm)

	A	B	C
2XL	30	92	112
3XL	33	93	124
4XL	33	95	132

Composition
100% polyester mesh
Polyester webbing 40 mm - 1450 DaN
PE foam 50 kg / m³
40 mm polyester straps

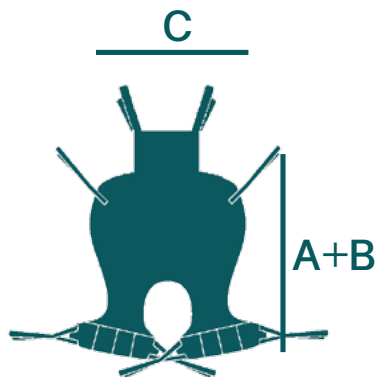

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

- Lateralize the patient. Fold the hammock in 2 along the vertical axis.
- Position the fold along the vertical axis in the middle of the patient's back.
- Place half of the strap on the free side of the patient.
- Fold the side of the strap on the side in support of the patient.
- Turn the patient on the back with half of the strap on the free side.
- Relateralize the patient on the strap on the previously free side.
- Remove the previously folded part and spread it out.
- Put the patient on his back on the deployed strap.
- Hang the high and low lines in the middle of the LP.

SETTINGS

The hanging system, multiple lines, allows to vary the position of the person.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 02.03/2019



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**