

*Prior to each use, it is important to check the status of the device:
seams, state of the straps and of the fabric.
Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.*


QUICK TOILET SLING



THIS MEDICAL DEVICE
MUST BE IMPLEMENTED
BY A TRAINED HEALTH
PROFESSIONAL



- Its widened U-shape completely clears the patient's buttocks, thus facilitating positioning in the toilet and improving the positioning and removal of the strap.
- Leg passage is from the outside to the inside.
- This strap prevents constriction of the hips when used uncrossed.

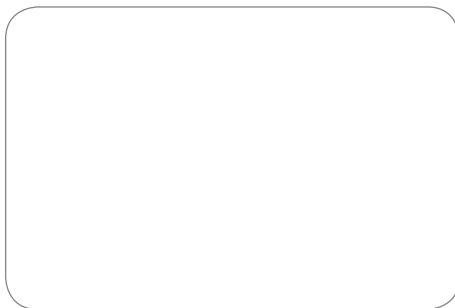

STO-PF-XL

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient


These products must not be in direct contact with a wounded skin.

QUICK TOILET SLING

SIZES & DIMENSIONS (in cm)

	A	B	C
2XL	132	30	134
3XL	132	30	144
4XL	138	32	156

Composition
100% polyester mesh
Polyester webbing 40 mm - 1450 DaN
PE foam 50 kg / m³
40 mm polyester straps

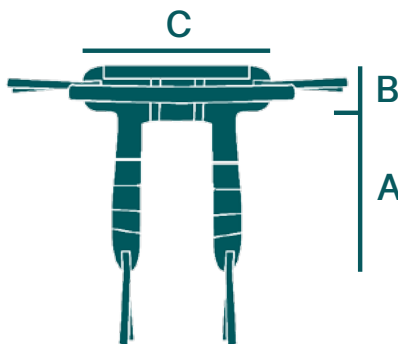

INSTRUCTIONS FOR USE

DO NOT USE THIS SLING IF THE SIZE IS NOT SUITABLE FOR THE PATIENT.

The backrest of this type of strap, is constituted by the wide part of the strap.

The thigh supports, meanwhile, are constituted by the branches of the U.

This type of strap has a system of attachment to the flail of the lift in four points:

1 at each end of the thigh supports.

Position the bottom of the backrest at the level of the shoulder blades so that the ears of the system of attachment are on both sides of the person. Pass the fasteners of the backplash under the arms of the person, so that the beads are under the armpits. Close the plastron.

While slightly lifting the right leg of the person, pass from the outside of the thigh inward the attachment of the right thigh support, proceed similarly for the support of left thigh. Hook the two backrest fasteners and the two thigh support clips to the flail of the lift.

2, on both sides of the backplash

SETTINGS

The attachment system, by multiple ears, allows to vary the sitting position of the person.

CAUTION

Read the instructions for use before using the sling.

Hang to the spreader bar the loop which are at the same level.

Do not change any adjustment while the patient is hanging in the sling.

Before lifting a patient, ensure that the sling is correctly hooked on the spreader bar.

v 02.03/2019



**THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.**