



This medical device is intended for patient positioning.

This is not a restraint system.

Before use, it is important to check the status of the device: stitching, state of the buckles, slings, fabric. Any damage will either be repaired by NAUSICAA Medical, or will result in a disposal.

FOOT SUPPORT




**THIS MEDICAL DEVICE MUST BE
IMPLEMENTED BY A TRAINED
HEALTH PROFESSIONAL**



**TO USE ONLY FOR THE INDICATED USE :
KEEPING YOUR FOOT ON
THE WHEELCHAIR PALLET**

Description

Keeps the foot on the pallet of the chair (with self-gripping attachment under and on the sides of the pallet).

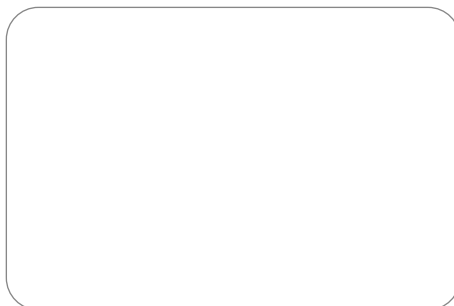

LMDP-HP-PF

v 05.12/2020

Important Recommendations :

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.



FOOT SUPPORT

HOSPITAL RANGE
Polyurethane-coated jersey lined with
polyurethane foam
Bias 100% polyester
Self-gripping 100% polyester
PU foam 100 kg / m³

INSTRUCTIONS FOR USE

to use only for the indicated use:

KEEPING YOUR FOOT ON THE WHEELCHAIR PAL- LET

- Attach each foot support to each wheelchair paddle with the self-gripping sole of the foot support.
- Place each foot of the patient in a footrest.
- Close the footrest on the patient's foot with the self-gripping device.



v 05.12/2020



THIS MEDICAL DEVICE MUST BE SET UP BY A PROFESSIONAL
OR THE USER MUST BE EDUCATED TO SET IT UP.