

## STAND-UP LIFT: BLUE WAYUP PLUS with electrical opening legs



MADE FROM STEEL  
WEIGHT CAPACITY : 150 kg  
CLASS 1 MEDICAL DEVICE

CONFORM TO STANDARD NF EN ISO 10535 : 2007

# Contents

## User Manual / Stand-Up Lift: BLUE WAYUP PLUS with electrical opening legs

• Intended Purpose Scope of Delivery .....	1
• Assembling Instructions .....	2-3
• Instructions for Use .....	4
• Stand-up Lifts Use .....	5-7
• Motorisation Informations .....	8
• Users Guide .....	9
• Slings Use .....	10
• Reuse Labelling .....	11-13
• Preventive Maintenance & Security Controls .....	14-15
• General Safety Guidelines .....	16
• Technicals & Dimensionals Characteristics .....	17
• Spare Parts .....	18
• Cleaning & Maintenance .....	19
• Motorisation Technicals Characteristics .....	20-21
• Troubleshooting Guide .....	22
• Warranty .....	23

**Blue**  
**wayUP** Plus

**NAUSICAA**  
Médical

# Intended Purpose

The stand-up lift BLUE WAYUP PLUS with electrical opening legs is an active class I medical device according to the EU Council Directive 93/42/EEC Appendix IX. It can be employed in all areas of care work and is intended for use by caregivers properly instructed in its operation.

It is intended for short-term use not involving contact with injured skin and requires the use of a suitable sling. The stand-up lift BLUE WAYUP PLUS with electrical opening legs may be used for patients weighing up to a maximum weight of 150 kg.

The stand-up lift BLUE WAYUP PLUS with electrical opening legs is only suitable for driving short distances within the patient's residence or area of activity on one floor. It must only be used on an even, horizontal floor. The climatic conditions must be an ambient temperature of 0°C to 40°C, humidity of 20% to 80%, air pressure of 700 hPa to 1060 hPa and in the range of normally composed atmospheric air. It is permitted to be used in wet areas. For example, this includes the toilet or the bath. This does not include using the patient hoist under the shower.

The stand-up lift BLUE WAYUP PLUS with electrical opening legs is not explosion protected and is not permitted to be used in potentially explosive areas.

Clinical pictures such as osteogenesis imperfecta, osteoporosis or spinal column damage, as well as mental disorientation or epileptic seizures can represent contraindications.

The stand-up lift BLUE WAYUP PLUS with electrical opening legs is only permitted to be used after careful consideration of the individual patient by the doctor and the care personnel.

## Standard stand-up lift

In the standard stand-up lift version the BLUE WAYUP PLUS with electrical opening legs is solely to be used for lifting and transferring a patient in a suitable sling. This is usually performed with the patient in a seated position.

## Active stand-up lift

In the active stand-up lift version, the BLUE WAYUP PLUS with electrical opening legs is solely to be used to assist a patient from a seated position to a standing position and then transfer them in a suitable sling. It serves as an aid to the natural movement of standing up and sitting down and can be driven with the patient standing up so that they may still go to the toilet independently for example. Additionally, the patient's participation stimulates its residual mobility by strengthening his circulation and musculature.

The use of the active stand-up lift requires the active participation of the patient. The ability to hold on to the lifting arm tightly is particularly important. Missing limbs or functional limitations of the musculoskeletal system (e.g., paraplegia) can limit or rule out the use of an active hoist. This must be decided by the operator on a case-by-case basis.

# Scope of Delivery

The stand-up lift has already been inspected at the factory to ensure that it is free from defects and that nothing is missing. Nevertheless, please check the product immediately after receipt for any damage which may have occurred during transport.

Use the delivery note to check that all items are present and that the delivery is thus complete.

Box	Contents	Units
BLUE WAYUP PLUS with electrical opening legs	BLUE WAYUP PLUS with electrical opening legs	1
	Battery pack (removable)	1
	Handcontrol	1
	User Manual	1

# Assembling Instructions

BEFORE USING YOUR STAND-UP LIFT, IT IS NECESSARY TO CHECK :

- The wheels turn and roll normally.
- The rear wheels function correctly.
- There is no wear or deformation on the hooks.

**WARNING :**

- To preserve the inserts, do not overtighten.
- Assembly is done with the brakes locked.

**Step 1:**

- Remove the mast from the box (the mast is composed of an 8000 N actuator + a lifting arm + a handlebar)
- Remove and unwrap the shin-block from the box
- Remove the base from the box (the base is equipped with anti-slip)

1



**Step 2:**

- Insert the mast in the base
- Screw a M8 bolt in each hole of the base
- Put the nuts covers

2



# Assembling Instructions

---

3

Step 3:

- Position the shin-block
- Screw the M8 hand wheel on the side

Your stand-up lift is ready for use.



# Instructions for Use

- The stand-up lifts are useful and effective when a patient has a certain degree of dependence coupled with a caregiver need for basic care.
- They are designed for patients whose mobility is reduced but who can however be supported temporarily on at least one leg (a minimum of tonic by the patient is required), and who require mechanical assistance to be put into standing and be moved.
- The stand-up lifts contribute to the stimulation of the patient and its mobility; stimulate the cardiac system; combating against osteoporosis and all disorders associated with immobility such as falls during transfers; stimulate brain activity thus favouring maintenance of continence; improves intestinal activity and bladder function.



- Using a stand-up lift requires a capacity assessment of the patient.
- When using a stand-up lift, patients can be categorized in 2 categories:
  1. Low-tonicity patients
  2. Medium-tonicity patients
- How to properly use a stand-up lift (1.):
  - Low-tonicity patients shall be transferred in a "semi-seated" position



# Stand-up Lifts Use

- How to properly use a stand-up lift (2):
  - Medium-tonicity patients shall be transferred in a standing position



- Furthermore, the handlebar must be used to push and/or pull the device with the hands on the upper part or on the sides, not below pushing upward.



- Likewise, the patient's shins must be in contact with the shin block at all times.
- In all cases, the caregiver must be properly trained for the safe handling of the device.



## DIRECTIONS FOR USE :

- Your stand-up lift is designed to lift people, do not use it for other purposes.
- Check that the patient's weight does not exceed the maximum weight that can be raised by the stand-up lift.
- Do not force the controls and features of the stand-up lift, they are all easy to use and do not require strength.
- Operate the stand-up lift by pushing the handles, never by pushing the patient.
- The stand-up lift must be handled with care when a patient is transferred and at a speed appropriate to the situation.
- Operate the stand-up lift on flat and smooth surfaces. It is not recommended to be used on a slope of more than 5°. If you are forced to move on a ramp, it is recommended that a second person helps you. Do not use an electric stand-up lift in a shower.
- Do not recharge the batteries near a bath or shower.

# Stand-up Lifts Use

---



## Parking brake

The parking brakes are one of the important safety elements of an stand-up lift. The rear castors are fitted with parking brakes which act directly on the wheels. To lock the castors, press the parking lever to the stop using your foot (Castor Locked). The stand-up lift can no longer be pushed in this setting. To release the castors, press the parking lever pointing upwards (Castor Released).

When locking the castors always insure that both castors are locked.  
If only one castor is locked on a sloping floor, the unlocked wheel will roll downhill around the locked wheel.

Depending on the slope of the surface, there is the danger that the stand-up lift will tip over sideways.

Castor Released



Castor Locked





# Stand-up Lifts Use



Before use, check that the actuator and the hand-control are correctly connected to the control unit (page 22)

Check the status of the control box and the battery

1



Check that the actuator is properly secured

2



Check that the battery is charged



# Motorisation Informations



# User guide

---

Choosing the right sling size is important in order to achieve the highest possible patient comfort and safety.

An assessment of the patient capacity is required before using a stand-up lift and more precisely to make sure that the patient can bear weight on at least one leg and has sufficient tonicity in the lower and upper parts of the body.

Additional parameters should also be taken into account depending on the situation, such as the patient's physical handicap, pathology and morphology.

Our R&D department has put in place a size colour-code in order to quickly identify the size of each sling:

Size	Color code	Plastron
S	Yellow	70 / 90 cm
M	Red	90 / 115 cm
L	Green	115 / 140 cm
XL	Dark Blue	140 / 165 cm
TU (One size fits)	Black	

In order to choose the right sling, the caregiver shall take into account the patient's waist size.

All our slings provide sustainable solutions for reassuring transfers and are compatible with all stand-up lifts with a 2-hook system.

---

## CAUTION

### IMPORTANT RECOMMANDATIONS

In order to get a maximum efficiency of these sort of product, it is necessary :

- to choose the appropriate size for the patient
- to get the best possible adjustment to the patient

These products must not be in direct contact with a wounded skin.

Read the instruction manual before using the slings.

The backrest hangers must be hung on the same color.

Never change the attachment setting of the ears to the stand-up lift hooks, when someone is installed in the sling.

Before lifting a person with the stand-up lift, make sure the sling is securely hooked to the hooks.

# Slings Use

## INSTRUCTIONS FOR USE

Position the bottom of the backrest in the lumbar region in order that the ears of the attachment system are on either side of the person.  
Place the fasteners of the backrest under the arms of the person.  
Close the plastron.  
Hook the two fasteners of the backrest to the stand-up lift hooks.

### SETTINGS

The fastening system, with multiple hangers, allows to adjust the position of the person.



## Transfer in sitting position for ALL PATIENTS



Position the "under thighs" strap and close it adjusting it without tightening too much.

Handlebar in lower position:  
Position the **yellow loops** on the central hooks of the handlebar.

Operate the actuator until halfway.

## Standing up for PATIENT WITH CHEST TONICITY



Handlebar in lower position:  
Position the **green loops** on the central hooks of the handlebar.

Operate the actuator until it stops.




# Reuse

The stand-up lift is suitable for repeated use.

The stand-up lift must be cleaned, disinfected and serviced according to the Preventive Maintenance and Security Controls (pages 14-15) at the very latest just before the reuse.

## Labelling

Labelling on the box

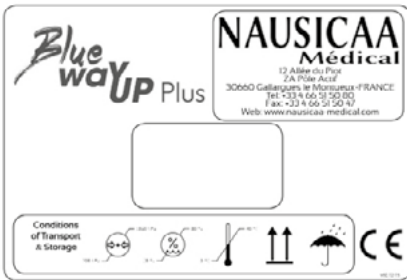

# Labelling



## SAFETY INSTRUCTIONS BEFORE EACH USE

**NAUSICAA**  
Medical

1

- Check the perfect condition of the hoist sling.
- Check the perfect condition of the traction sling (EASYLEV, NAUSILEV).
- Verify the presence and conditions of safety devices: all axis and their constrained fixations (pins, clips and bolts) and the tightness of the screws (see user manual).

DO NOT USE THE DEVICE IN CASE OF ANY LACK OR BAD OVERALL STATE

**Blue  
wayUP** Plus

**NAUSICAA**  
Médical

2

POIDS MAXIMUM  
MAXIMUM WEIGHT  
MAXIMALE TRAGFÄHIGKEIT  
PESO MAXIMO  
PESO MASSIMO

**150 kg**

CE **NAUSICAA** FRANCE  
Médical v 02.12-15

3



## BATTERY MAINTENANCE

**NAUSICAA**  
Medical

A POORLY MAINTAINED BATTERY DETERIORATES

- Regularly recharge the battery at the maximum.
- Making small incomplete charges shortens the battery life.
- The battery should not be exposed to temperatures neither too low nor too high.
- The batteries status depends on the maintenance and therefore can not be under warranty.
- It is important to check the charger if the battery no longer works.

4

**Blue  
wayUP** plus CE

BWPL-0036-FM



Année/Mois de Fabrication: \_\_\_\_\_ / \_\_\_\_\_

Capacité Maximale : 150 kg

NAUSICAA MÉDICAL  
11 Avenue de Paris - 93110 La Plaine Saint-Denis  
93000 La Plaine Saint-Denis (France)

5

# Labelling

---



# Preventive Maintenance & Security Controls

Stand-Up Lift: BLUE WAYUP PLUS with electrical opening legs

Serial Number :

Control of security points (Frequency: depending on use, recommended every three months)  
This is a sight control : any wear trace must lead to a replacement.

		GOOD	WRONG	DATE OF CHANGE
1	Connection head of the actuator (bolted)			
2	Connection base of the actuator (bolted)			
3	Connection head of the mast (bolted)			
4	Locking base of the mast			
5	Fixing of the castors (bolted)			
6	Sliding of the sub-patellar support (adjustable knob)			
7	State and sticking of the anti-slip material			
8	Fixing of the mast handle			
9	Overall state of the device (welds, joints, oxidation)			

Control of electrical functions (Frequency: depending on use, at least once a year)  
this control must be carried out when the hoist is loaded.

		GOOD	WRONG	DATE OF CHANGE
A	Operation of the actuator			
B	Operation of the emergency system			
C	Batteries status (charge holding)			
D	Handcontrol status			
E	Control operation on the battery block			

Inspection carried out on :	Inspection carried out by :	Date of the next inspection :




# Preventive Maintenance & Security Controls

Control of security points (Frequency: depending on use, recommended every three months)  
This is a sight control : any wear trace must lead to a replacement.

- 1 → Connection head of the actuator (bolted)
- 2 → Connection base of the actuator (bolted)
- 3 → Connection head of the mast (bolted)
- 4 → Locking base of the mast
- 5 → Fixing of the castors (bolted)
- 6 → Sliding of the sub-patellar support (adjustable knob)
- 7 → State and sticking of the anti-slip material
- 8 → Fixing of the mast handle
- 9 → OVERALL STATE OF THE DEVICE (WELDS, JOINTS, OXIDATION)

THESE CONTROLS CAN BE CARRIED OUT WITHOUT LOAD

→ Periodical Greasing

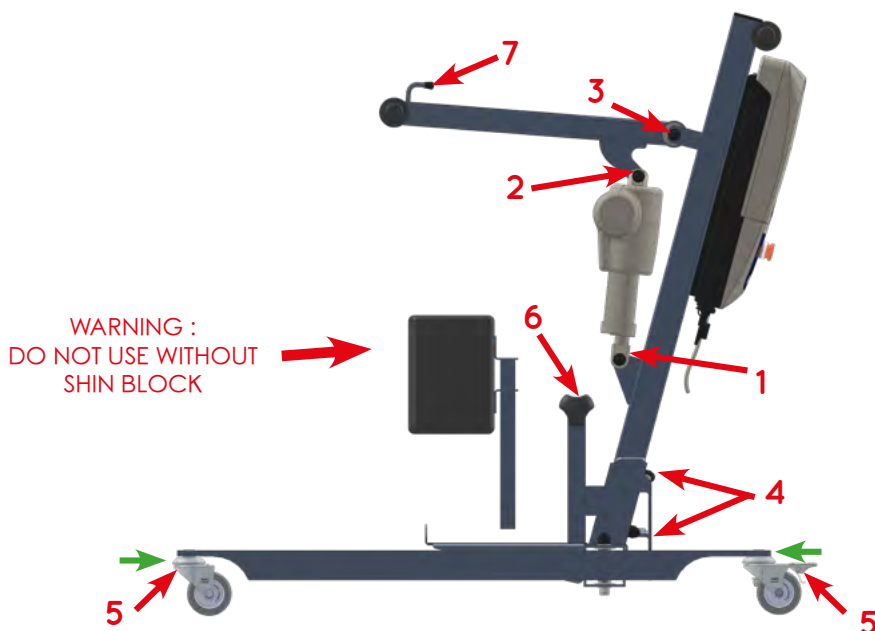

# General Safety Guidelines

---

1. Use the Stand-Up Lift only for its intended purpose, in accordance with the legislation for medical devices, the regulations for labor protection and accident prevention, as well as the generally recognized rules of technology.
2. Note that the Stand-Up Lift is a medical device, therefore the user is required to comply with the directive on the use of medical devices.
3. The requirements for the electrical installation of the room or area where the Stand-Up Lift is used must meet the current state of the technology.
4. Using the Stand-Up Lift only after being trained for its handling.
5. Before using the device, please read the complete User Manual in order to avoid damage due to improper handling or exposure to risks. The User Manual contains important information and notes required for the Stand-Up Lift use.
6. Use the Stand-Up Lift in accordance with this User Manual only. Keep the User Manual for future reference in case of questions. Join this User Manual to the Stand-Up Lift in case of change of ownership.
7. Before any use, it must be ascertained whether the Stand-Up Lift and its accessories are in working order and in impeccable conditions.
8. Before using the Stand-Up Lift with other medical and non-medical devices, check that the combination of these products is allowed and can be used together safely.
9. Assembly, commissioning, maintenance and repair of the Stand-Up Lift should be entrusted only to specialists.
10. It is up to the user or operator to ensure (through adequate measures and instructions) that mechanical stress of the charging cord (through bending, tension, shear, crushing) is excluded during loading or cleaning of the part. This also applies to the electrical cables of other devices used with the Stand-Up Lift.
11. Respect the activation duration and the maximum weight capacity. These values must not be exceeded, otherwise the Stand-Up Lift safe operation is no longer guaranteed.
12. Do not expose the Stand-Up Lift to direct sunlight or heat.
13. Ensure that no moisture enters the electrical system.
14. Avoid mechanical stress on electrical cords. Pull, bend or crush the cords may damage them.
15. Charge batteries in a well ventilated area.
16. Electromagnetic interferences can not be excluded from the Stand-Up Lift and other devices. If there is a risk of such interference, the source of interference must be removed or the Stand-Up Lift should not be used.
17. Disruptions caused by the use of portable communication devices can not be completely excluded. This is why a safe distance of at least 3 meters must be kept in order to ensure the functioning of the Stand-Up Lift safely.
18. Do not leave unattended children stand near the Stand-Up Lift.
19. The Stand-Up Lift should not be used as soon as abnormal noises, damages or any other malfunctions occur. In this case, do not connect the wall charger to the Stand-Up Lift and inform us.
20. If damaged or defective, the Stand-Up Lift should not be used and should not be connected to the direct current. Inform the dealer to remedy the default or failure.


# Technicals & Dimensionals Characteristics

- Frame made in steel
- Laser cutting and bending sheet metal, tubes and profiles
- Epoxy baked painting
- Ball bearing castors
- Total weight with electrical opening: 35 kg
- Weight of the heaviest part - BASE : 6 kg
- Weight capacity : 150 kg
- Lifespan : 8 years (except electrical parts)
- \* SCP : Suspension central point



## Dimensionals Characteristics

- A. Total length: 100 cm
- B. Minimum lifting height: 84 cm
- C. Maximum lifting height: 128 cm
- D. Total height: 100 cm
- E. Frame height: 11 cm
- F. Minimum base width: 57 cm
- G. Maximum base width: 90 cm
- H. Turning diameter: 115 cm

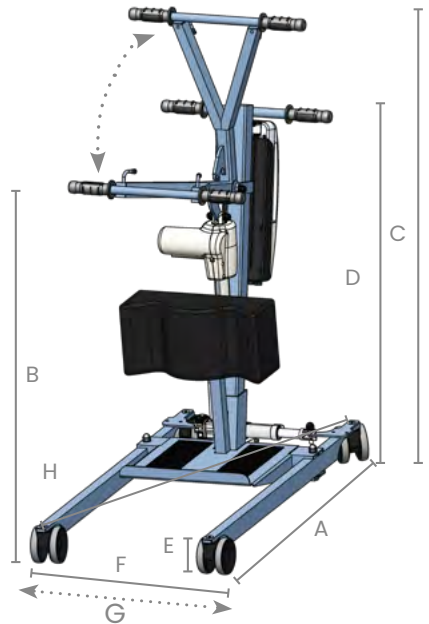

# Spare Parts

## BLUE WAYUP PLUS WITH OPENING LEGS

FRAME	
WUW-EBS	LIFTING ARM FOR BLUE WAYUP
WUW-WT	WAST FOR BLUE WAYUP
WUW-CTR	SHIM BLOCK SHORT SUPPORT (without foam)
WUW-EP-PGCHN	LEFT FOOT FOR BLUE WAYUP DBL AND BLUE FLYER COMPACT
WUW-EP-PGRCH	RIGHT FOOT FOR BLUE WAYUP DBL AND BLUE FLYER COMPACT
WUW-EP	SPREADING SYSTEM SET FOR BLUE WAYUP (without pedal)
WUW-EP-PD	FOOT PEDAL BLUE WAYUP DBL
WUW-EP-EMB	BASE FOR BLUE WAYUP DBL
WUW-EP-PLA	REMOVABLE PLATFORM FOR BLUE WAYUP DBL
WUW-PGN	LIFTING ARM GRIP HANDLES (set of 2)
WUW-MASR	SUB-PATELLAR FOAM SUPPORT FOR STAND-UP LIFT
WUW-AJADZ	WOM-SHD (2 branches of 2 pieces, 2 right and 2 left)
WUW-RUNPD100	BLACK TWIN CASTORS WITH RED BRAKE DIAMETER 100mm (ball bearing)
WUW-RUND100	BLACK TWIN CASTORS DIAMETER 100mm (ball bearing)
WUW-VPS	BLUE PIGEON STICKER PAINT
WUW-SPSP	BLUE PIGEON SPRAY PAINT
WUW-AG-SLK	WADEHMAN - UNIVERSAL LEG STRAP
INFORMATION VECBOX 3 FOR 2 ACTUATORS (electrical opening legs)	
VS-TLCH3-EDP	4 FUNCTIONS REMOTE CONTROL FOR VECBOX 3
VS-TLCH3-CCG-EDP	CONTROL BOARD FOR 4 FUNCTIONS REMOTE CONTROL FOR VECBOX 3
VS-TLCH3-CRD-EDP	CORD FOR 4 FUNCTIONS REMOTE CONTROL FOR VECBOX 3
VS-AMTL	MAGNET FOR REMOTE CONTROL
VS-CTRL	REMOTE CONTROL HOOK
VS-TLCH3-EDP-FA	FRONT LABEL FOR REMOTE CONTROL 4 FUNCTIONS FOR VECBOX 3
VS-BCAM	REMOVABLE BATTERY PACK (with 2 internal batteries 5A/12V) WITH LOAD DISPLAY AND BALANCING
F3-BT2IN	SET OF 2 INTERNAL BATTERIES 5A/12V
BAI-LI	INTERNAL LITHIUM BATTERY 5A / 24V
VS-BCAM-LI	LITHIUM BATTERY PACK WITH DISPLAY
VS-BCAM-BORT	2 ELECTRONIC REMAINS SET (display and balancing) FOR BATTERY PACK VS-BCAM
VS-BCAM-BCFC	CONNECTION KIT WITH FUSES FOR BATTERY PACK VS-BCAM OR VS-BCAM-SMP
FUS-5-2B-200MA	INTERNAL DELAY FUSE 500-200MA (set of 5)
FUS-5-2B-1A	INTERNAL DELAY FUSE 5 a 2B-1A 250 v (set of 5)
VS-BCAM-CAL	BATTERY PACK COMPLETE CASING
VS-BCAM-FRE	BATTERY PACK ADHESIVE FRONTAGE DISPLAY FOR VS-BCAM
VS-BCAM-CIL	BATTERY PACK HOOK FASTENING
VS-BCC3-EDP	CONTROL BOX VECBOX 3 4 FUNCTIONS
VS-BCC3-CCG-EDP	CONTROL BOARD FOR VECBOX 3 4 FUNCTIONS
VS-BCC3-BA	CONTROL BOARD WITH DISPLAY FOR VECBOX 3 2 FUNCTIONS
VS-KONF	CONNECTION KIT WITH FUSES FOR CONTROL BOARD VS-BCC-CCG AND VS-BCC-CCG-EDP
VS-BALR	COMPLETE EMERGENCY STOP BUTTON
VS-BCH	CONTROL BOX POWER CORD INPUT PLUG
VS-CRD	POWER CORD
VS-EIL	POWER CORD REMINDER HOOK (set of 2)
VSUP-VRM	800N LIFTING ACTUATOR LIFTING STROKE 110mm
VS-WP-VRM-CHT	HEAD SCREED FOR WAY UP ACTUATOR
VS-VIRM-B-CEMB	800N LIFTING ACTUATOR BASE SCREED (all models)
VS-VIRM-B-CAN	800N (all models) AND 3000N [VSMB-VIRM] LIFTING ACTUATORS COMPLETE CASING
VS-VIRM-B-B-CRD	800N AND 3000N LIFTING ACTUATORS CORD (all models)
VSVM-ECF3	3000N OPENING BASE USES GREY ACTUATOR
VS-VIRM-ECF3-CHT	3000N OPENING BASE USES GREY ACTUATOR HEAD SCREED
VS-VIRM-ECF3-CEMB	3000N OPENING BASE USES GREY ACTUATOR BASE SCREED
VSVM-ECF3-CRD	3000N OPENING BASE USES GREY ACTUATOR CORD
VS-CMB	WALL CHARGER FOR BATTERY PACK

# Cleaning & Maintenance

---

## BEFORE ANY HANDLING:

- Disconnect the AC power cord.
- Check that all electrical components are connected to each other.
- Clean electrical enclosures of the actuators and of the handsets, if they have been soiled by body fluids, especially urine.

## OBJECTIVE:

- Repair the stand-up lift and prevent the transmission of germs from one patient to another.
- Remove any organic soil by mechanical action (cleansing) or chemical action (disinfection).

## CLEANING PROCESS:

- Clean the surfaces with a wet cloth by using an appropriate detergent.
- A regular cleaning is recommended.

## WARNING:

- The detergents used must have a neutral pH.
- Avoid the abrasive products and solvent, they could damage the device surface state.

## DAILY MAINTENANCE:

- By means of a product applied on the surface in one operation.
- After the patient's discharge by respecting the following operation:
  - Cleaning is carried out by means of a cloth impregnated with a disinfectant solution surface.
- Maintenance by specific service providers after removal of the stand-up lift from the facility:
  - Biological cleaning operation
  - Steam cleaning of various flat surfaces. Change surfaces regularly when washing to avoid water load. Steam cleaning of inaccessible areas. For tubes, steam use with a microfiber mop. Do not direct steam directly on the electrical boxes.

## CAUTION:

- Disinfection of actuators, electrical boxes and handsets with a microfiber cloth impregnated with disinfectant.

## MONTHLY MAINTENANCE:

- Check that the device does not present any visible damages.
- Check that no part is missing.
- Check the good functioning of the castors and that no matter prevents their bearing.
- Check the good functioning of the commands as well as the connections of the remote control and the actuator on the battery.
- Clean the sockets and the commands buttons with a dry cloth, if needed with a wet cloth.
- Check the good state of the electric cables.

Never clean the stand-up lift, especially the electrical system, with a high pressure cleaner, water hose or similar because doing so could damage the surfaces and seals, and water could enter.

# Motorisation Technicals Characteristics

- Removable battery pack
- Digital display of the autonomy
- Emergency stop
- Electrical security lowering system
- Wall charger (in option)
- Remote control 4 functions
- Conform to Standard EN 60601-1
- DC motor low voltage 24V
- 24V / 120VA power
- Maximum strength : 8000 N
- Electrical opening actuator power : 3000 N
- Lifting stroke : 11 cm
- Electronic protection when overloaded
- Protection class : II
- Protections types :
  - Remote control: IP54
  - Control box: IP65
  - Battery pack: IP65
  - Actuator: IP55
- Stop limit switch
- Box in ABS
- Handcontrol with spiral cable, low voltage 24V
- Protection against overloading through thermo-switch
- Loudness : less than 55dB at a distance of 1 meter.
- Electromagnetic compatibility : conforms to EN 60601-1-2



Device  
class II



## Battery Charging Diagram :



## A. LCM DISPLAY

### 0. Information display

- To view the information registered in the control box, you must press the up and down buttons of the box for 5 seconds.

### 1. Operating factor

- Definition: It accounts for the actuator (up / down) any breach of the duty cycle = non-respect of the actuator rest.



### 2. Overloading of the actuator / a

- It counts the number of times an attempt to lift a person weighing beyond the maximum weight limit is performed. When it occurs, the actuator will stop and an anvil will appear, it will be impossible to continue.



### 3. Overloading of the actuator / b

- It counts the number of times an attempt to spread or close the opening legs while they are stuck is performed. When it occurs, the actuator will stop and an anvil will appear, it will be impossible to continue.



### 4. Incomplete load

- It records all incomplete loads below 24,6V. This corresponds to the number of charges damaging the battery.



### 5. Time of use

- Definition: It displays the cumulative operating time of the device. Starting from 600,000 seconds of use, a maintenance key is displayed; it is then essential to carry out a complete control of the device.



# Troubleshooting Guide

Problem	Origin	Solutions
The product, without patient, does not go down.	Our products need some weight to go down.	Push down on the lifting arm while pushing the lowering button on the remote control.
Mobile parts are too hard, product is difficult to handle.	Mobile parts need to be greased.	Grease the mobile parts.
The actuator does not work but the user can hear a "click" in the control box when the remote control button is pushed.	<ol style="list-style-type: none"> <li>1 . Batteries are unloaded.</li> <li>2 . The actuator jack is not plugged.</li> <li>3 . The actuator jack is deteriorated.</li> <li>4 . There are damages on the actuator or on the control box.</li> </ol>	<ol style="list-style-type: none"> <li>1 . Load the device.</li> <li>2 . Plug the actuator jack.</li> <li>3 . The actuator has to be repaired.</li> <li>4 . The entire electric system has to be checked.</li> </ol>
The actuator does not work and the user does not hear a "click" in the control box when the remote control button is pushed.	<ol style="list-style-type: none"> <li>1 . Emergency button is activated.</li> <li>2 . There is no contact between the battery and the control box.</li> <li>3 . Batteries are unloaded.</li> <li>4 . The remote control jack is not plugged.</li> <li>5 . The remote control jack is deteriorated.</li> <li>6 . There are damages on the remote control or on the control box.</li> </ol>	<ol style="list-style-type: none"> <li>1 . Turn the button to unlock the device.</li> <li>2 . Take off the battery and put it back.</li> <li>3 . Check the charging cord.</li> <li>4 . Plug the remote control.</li> <li>5 . Change the remote control.</li> <li>6 . The entire electric system has to be checked.</li> </ol>
The actuator stops when lifting.	<ol style="list-style-type: none"> <li>1 . Batteries are unloaded (sound system + "low battery" written on the screen).</li> <li>2 . The product has been used for too long.</li> <li>3 . The weight lifted is too heavy.</li> </ol>	<ol style="list-style-type: none"> <li>1 . Check the charging cord then load battery.</li> <li>2 . Let the device rest a moment (as a protection for the actuator lifespan).</li> <li>3 . Reduce the weight lifted.</li> </ol>
The actuator does not work at all	<ol style="list-style-type: none"> <li>1 . The control box is broken down.</li> <li>2 . The remote control is broken down.</li> </ol>	<ol style="list-style-type: none"> <li>1 . The control box has to be changed.</li> <li>2 . The remote control has to be changed.</li> </ol>



# Warranty

---

• Article 1 : NAUSICAA Médical S.A.S. warrants this product against any defects in manufacturing and assembly of mechanical and electronic components. This warranty is for devices used only in accordance with NAUSICAA Médical S.A.S. terms of use.

The warranty covers all mechanical and electrical parts, except battery and breakage. This warranty whose terms of use are defined below is valid for 60 months from the date of first departure from NAUSICAA Médical S.A.S.

• Article 2 : The warranty entitles the free labor and the free replacement of defective parts.

• Article 3 : The original out-going shipping costs of the device and all associated costs are the responsibility of the distributor. The goods always travel at the risk and the responsibility of the distributor.

Under warranty : return costs after intervention will be borne by NAUSICAA Médical S.A.S.

Out of warranty: return costs are at the expense of the distributor whether or not he accepts the repair estimate.

• Article 4 : The warranty does not apply if the claims are consecutive to:

- accident, misuse or neglect of the unit by the end customer.
- shipping performed without adequate protection.
- alteration or transformation not validated by NAUSICAA Médical S.A.S.
- the impact of external elements (natural disaster, fire, shock, humidity, flood, lightning, ...).
- installation and/or use in a non-compliant way with technical standards and safety (in case the unit would work in a country other than the country of origin) ; and/or if the electrical power is not suited for the operating voltage of the device.
- a lack of routine maintenance.

• Article 5 : The distributor may not invoke the benefit of the warranty:

- if the device serial number has been removed, altered or rendered illegible.
- if the device under warranty has been modified without the approval of NAUSICAA Médical S.A.S.

• Article 6 : During the repair of any defective equipment, no loan will be made.

• Article 7 : All warranty claims must be exercised through the distributor.

• Article 8 : Sending spare parts under warranty will be made after consultation with the distributor customer service.

• Article 9 : Defective parts changed under or out of warranty will be guaranteed for 6 months from the date of repair or sending of the spare parts.

• Article 10 : No distributor can unilaterally change the terms of this warranty.





# NAUSICAA Medical

## CONTACT

Phone: +33 (0) 466 717 184

Fax: +33 (0) 466 532 305

Email: [export@nausicaa-medical.com](mailto:export@nausicaa-medical.com)

Website: [www.nausicaa-medical.eu](http://www.nausicaa-medical.eu)

Manufactured by NAUSICAA Médical S.A.S.



**NAUSICAA**  
Medical

12 Allée du Piot - ZA Pôle Actif  
30660 Gallargues le Montueux  
FRANCE